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Model PO400AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to stabilise and support the cervical spine during travel by various modes of transport, such as plane, car, bus or train. Thanks to its ergonomic croissant-type shape, the cushion envelops the neck, providing a comfortable sitting position, reducing muscle tension and fatigue and preventing neck pain. Made from skin-friendly materials, it is lightweight, compact and easy to carry, making it ideal for travellers requiring comfort and stable head and neck support for extended periods.

Intended use

Medical device designed to stabilise and support the cervical spine in adults during prolonged sitting, especially when travelling, to reduce pain, muscle tension and prevent neck strain.

Contraindications

Allergy to any of the product ingredients. If an allergic reaction occurs, discontinue use.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Ergonomically shaped like a croissant to fit the user's neck.
- Stretch fill for comfort and support.
- Cover in soft, skin-friendly fabric.
- Lightweight and compact design, easy to carry.
- Fastener or snap to keep the cushion firmly around the neck.
- Easy to clean and maintain.

First use

1. Remove the cushion from the packaging.
2. Shape it if it has been folded or compressed during transport.
3. Place the cushion around your neck, adjusting its shape to fit your neck.
4. Make sure the cushion fits firmly and does not put uncomfortable pressure.
5. Adjust the clasp if necessary.
6. When using for the first time, a short trial (a few minutes) is recommended to check the comfort and reaction of the skin.

Safety and usage rules

- Wear the pillow so that the thickest part of the pillow is under the chin - this provides better support for the head and prevents it from falling forward during sleep.
- Adjust the cushion to the shape of your neck, avoiding too much pressure and ensuring a comfortable fit.
- Do not use the cushion on damaged or irritated skin.
- Check the condition of the cushion regularly - do not use it if it is damaged.
- Only use the cushion for its intended purpose - to support the neck while sitting.
- Store the cushion in a dry and clean place after use.
- The product is OEKO-TEX Standard 100 certified, confirming the absence of substances harmful to health.

Maintenance

- The cushion cover can be washed by hand or in a washing machine at up to 30°C using a mild detergent and fabric softener; do not use bleach.
- Dissolve the detergent before adding it to the wash to avoid localised discolouration of the fabric.
- Wash separately from other garments to avoid staining.
- Do not soak the cover in cold water for more than 5 minutes.
- Dry on the left side, avoiding direct sunlight.
- After washing, it is advisable to stretch and hang the cover to dry; the ironing temperature should not exceed 110°C.
- The pillow insert (foam core) is not suitable for washing, dry-cleaning, bleaching or exposure to direct sunlight. Store in a dry and well-ventilated place, preferably in the original packaging, to prevent dampness and mould growth.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 2 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Batch/series number
	Model		Warning
	Date of production		Medical device manufacturer
	Unique identification code of the device		Keep out of sunlight
	Protect from moisture		
		Cover: Wash at up to 30°C, do not bleach, dry flat in the shade, do not iron, do not dry-clean.	

Pillow core: Do not wash, do not bleach, dry flat, do not iron, do not dry-clean.

Product content:

Knitted polyamide/elastane (also fleece), viscose/polyester/elastane, polyester/cotton/elastane and polyester/elastane, white and ready-made dyed (acid, reactive and disperse dyes), 100% polyester (also coral fleece), white and dyed with disperse dyes, partly with white polyurethane foam, including accessories (sewing thread, zip, plastic snap/clasp, printed and woven label, elastic band); Pillow core made of thermoplastic polyurethane foam (memory foam).

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ACTION S.A.