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Stopień schodek do wanny, PREMIUM

Model: SW100AM

Wstęp

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Model SW100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to facilitate safe entry and exit from the bathtub for people with reduced mobility, including seniors, people with disabilities and patients following injury or surgery. The step stool is made of durable, waterproof materials, has a non-slip surface and rubber pads on the feet to ensure stability and minimise the risk of slipping on wet surfaces. It can also be used as a footrest or as an aid for reaching higher places in the bathroom, and its compact size and lightweight design make it easy to carry and store. This product enhances the user's independence, improves comfort and safety during daily hygiene activities and, thanks to its versatile application, will work well in both home and medical settings.

Intended use

A medical device designed to facilitate safe entry into and exit from the bathtub for people with reduced mobility, providing stable support and minimising the risk of slipping during daily hygiene activities.

Contraindications

Do not use with imbalance, coordination disorders, weight over 130 kg, step damage or slippery ground.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Platform made of durable, waterproof plastic with non-slip properties, ensuring safety during use.
- Four height-adjustable legs made of powder-coated metal (steel / aluminium), which guarantees stability and durability of the structure.
- Rubber caps on the ends of the legs that prevent the step from sliding and protect the floor from scratches.
- Lightweight design, allowing the step to be easily carried and positioned anywhere in the bathroom.
- Easy-to-clean surface, resistant to water and cleaning agents, making it easy to maintain hygiene.

Assembly

- Remove all components from the packaging and make sure the kit is complete.
- Attach the legs to the platform by inserting them into the appropriate holes and tightening them.
- Set the desired height by adjusting the length of the legs and locking them at the desired level.
- Check the stability of the step stool by ensuring that all legs are evenly aligned and that the rubber caps sit well on the ground.
- Place the step stool in the desired location in the bathroom, preferably on a dry, flat surface, ready for use.

Safety and usage rules

- Before each use, check that the step stool is stable and that all legs are correctly attached and positioned on a level, dry surface.
- Make sure the rubber caps on the legs fit snugly and are not damaged.
- Do not exceed the maximum permissible load.
- Only use the step stool for its intended purpose - as an aid to getting in and out of the bath.
- Do not use the stepladder if you have a severe imbalance or coordination disorder without third-party supervision.
- Do not use the step stool if it is damaged or visibly cracked.
- After using the step stool in a damp environment, it is recommended to dry it and clean it regularly to prevent slipping and to maintain hygiene.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Wipe the step stool down dry after each use to prevent moisture build-up and deposits.
- Regularly check the condition of the rubber leg caps and adjustment elements - replace them with new ones if worn or damaged.
- Avoid prolonged contact of the step stool with water - do not leave it in the bath or on a wet surface for long periods of time.
- Store the step stool in a dry place, away from direct sunlight and heat sources.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

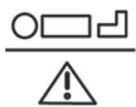
Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)

	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Platform: Durable plastic (polypropylene).
- Legs: metal (aluminium/powder-coated steel, corrosion-resistant).
- Leg caps: rubber or non-slip plastic to protect the ground and ensure stability.
- Adjustment elements: plastic and metal (for leg height locking).

Technical data

- Overall height: approx. 23 cm
- Platform dimensions: approx. 40 cm (length) x 28 cm (width)
- Height adjustment: yes
- Maximum load: 120 kg
- Colour: white.
- Weight: < 3kg

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ACTION S.A.