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Model: SP100AM

Wstęp

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Model SP100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

Medical device equipment (beds: medical / rehabilitation / nursing) intended for use in medical facilities. The table is equipped with a tilt and height-adjustable top, allowing comfortable use by the patient while eating, reading or writing. The sturdy steel construction on four castors allows the table to be easily moved and positioned at the patient's bedside, while locks on the castors ensure safety during use. The tabletop surface is made of a material that is easy to disinfect and resistant to mechanical damage. The table is designed with ergonomics, functionality and the hygienic requirements of the hospital environment in mind.

Intended use

A device designed to support the daily activities of patients with reduced mobility. Allows for meals, occupational therapy and bedside nursing procedures thanks to the adjustable height and angle of the tabletop. The mobile design on castors ensures easy movement within the hospital ward.

Assembly

1. Unpacking all table components and checking for completeness.
2. Assembly of the H-shaped base by connecting the steel elements.
3. Attach the four swivel castors to the base, making sure that the locking mechanisms work properly.

4. Installation of the vertical adjustment arm to the base, maintaining the correct screw tightening torque.
5. Installation of a mechanism for adjusting the height and angle of the tabletop.
6. Fixing the laminate top to the adjustment arm.
7. Checking the smooth operation of all regulatory mechanisms.

First use

1. Position the table on a level, stable surface with a minimum space of 45 cm around the unit.
2. Checking the operation of all castors and locking mechanisms before the first move.
3. Height adjustment of the tabletop according to the user's needs.
4. Setting the appropriate angle of the tabletop depending on the intended use.
5. Locking of the castors once the table is in the target position.
6. Carry out a stability test by gently loading the tabletop before handover.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible weight, per tabletop – 20 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

1. Before each use, check the condition of all moving parts, adjustment mechanisms and ensure there is no structural damage.
2. Once the table is in the target position, always lock the castors to prevent accidental movement during use.
3. Do not exceed the maximum permissible tabletop load to avoid structural damage or the table tipping over.
4. The table should only be placed on a level, stable surface with a minimum of space around the unit for safe manoeuvring.
5. All adjustments to the height and angle of the tabletop should be made slowly and carefully, making sure that the mechanisms are correctly locked in the chosen position.
6. It is forbidden to use the table as a ladder, a stand up support or in a way that is not in accordance with its medical purpose.
7. Do not use the table outdoors, on uneven ground or near heat sources, which can lead to dangerous situations.
8. Children must only use the table under adult supervision to prevent misuse.
9. The operation of all locking mechanisms, castors and the general condition of the table should be checked regularly.
10. The table should be cleaned and disinfected according to the procedures of the medical facility, using appropriate agents safe for the construction materials.

Maintenance

1. Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
2. Avoid the use of: strong detergents, abrasives, bleach, petrol and all solvents, as these may cause damage to the product, visible defects and lead to a loss of warranty.
3. The castors should be checked regularly for smooth running, cleaned of dust and dirt and the bearings lubricated as necessary.
4. The height and angle adjustment mechanisms of the tabletop should be lubricated periodically with suitable products and checked for correct operation.
5. All bolted connections should be inspected monthly and tightened if loose components are detected.
6. The surface of the tabletop should be protected from direct contact with hot objects and sharp tools that can damage the laminate.
7. The caster brakes should be tested regularly and, if necessary, adjusted or worn components replaced.
8. Store the table in a dry room, protecting it from excessive moisture, which can damage the metal components.
9. If cracks, deformation or other structural damage is detected, the table should be taken out of use immediately until repaired.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Product content

- The top is made of MDF board coated with melamine laminate, which provides a smooth, moisture-resistant and easily disinfectable surface.
- The edges of the tabletop are protected by an ABS plastic profile to protect the edges from mechanical damage.
- The vertical frame and table base are made of powder-coated carbon steel for high corrosion resistance and structural durability.
- Control elements, such as handles and knobs, are made of polypropylene (PP) or polyamide (PA).
- Transport castors are made of polycarbonate (PC) or polyurethane (PU) with a steel core.
- Bolts, nuts and other connecting elements are made of galvanised or stainless steel.

Technical data

- Tabletop dimensions: 60 x 38 cm
- Height adjustment: 72 to 118.5 cm
- Body mass: 6 kg
- Maximum permissible tabletop load: 20 kg

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use	 MAX 20 kg	Maximum permissible weight per tabletop (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

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ACTION S.A.