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Model PO500AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device intended for use between the knees while sleeping or resting in a lateral position. Its ergonomic shape ensures correct positioning of the legs, hips and spine, which helps to reduce tension in the hip and knee joints and relieves pain associated with strain in these areas. Made from thermoelastic memory foam, it adapts to the user's anatomy, providing comfort and support throughout the night. The cushion is particularly recommended for people with hip, knee and back pain, pregnant women, as well as for varicose vein prophylaxis and recovery after orthopaedic surgery. The soft fabric cover, fitted with a zipper, allows the product to be easily hygienically maintained.

Intended use

A medical device designed to maintain correct leg alignment during side sleeping, relieve pain in the knees, hips and lower back and support recovery after orthopaedic surgery.

Contraindications

Allergy to any of the product ingredients. If an allergic reaction occurs, discontinue use.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- The product is made from high quality thermoelastic memory foam, providing an optimal fit to the user's anatomy.
- The ergonomically shaped form provides stabilisation and comfortable support for the lower limbs.
- The cover is made of soft, pleasant-to-touch material and features a zip for easy removal and hygiene.
- The product is designed to be used while sleeping or resting in a lateral position.
- It ensures high quality workmanship and long service life.

First use

- Open the package and take out the orthopaedic pillow.
- Check the condition of the product and the cover for any damage.
- Place the pillow in the correct position between your knees, lying on your side, so that the contoured shape provides stable and comfortable support for your lower limbs.
- Make sure the cushion fits snugly against your legs without causing pressure or discomfort.
- Start using - the cushion requires no additional configuration.

Safety and usage rules

- Only use the pillow for its intended purpose as a medical device for cervical support during sleep.
- Wash the cover according to the manufacturer's recommendations before first use.
- Arrange the head in the middle of the pillow so that the neck is not overly bent or stretched; place the contoured section under the neck.
- Sleeping on the stomach is not recommended as it does not provide the correct support for the spine.
- Wash the cover regularly to maintain hygiene and prevent the growth of allergens and micro-organisms.
- Do not use the cushion if you have an allergic reaction to any of the ingredients.
- If discomfort or pain persists, consult your doctor or physiotherapist.
- Do not expose the cushion to high temperatures or chemically clean it to avoid damaging the shape memory foam.
- The product is OEKO-TEX Standard 100 certified, confirming the absence of substances harmful to health.

Maintenance

- The cushion cover can be washed by hand or in a washing machine at up to 30°C using a mild detergent and fabric softener; do not use bleach.
- Dissolve the detergent before adding it to the wash to avoid localised discolouration of the fabric.
- Wash separately from other garments to avoid staining.
- Do not soak the cover in cold water for more than 5 minutes.
- Dry on the left side, avoiding direct sunlight.
- After washing, it is advisable to stretch and hang the cover to dry; the ironing temperature should not exceed 110°C.
- The pillow insert (foam core) is not suitable for washing, dry-cleaning, bleaching or exposure to direct sunlight. Store in a dry and well-ventilated place, preferably in the original packaging, to prevent dampness and mould growth.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 2 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Batch/series number
	Model		Warning
	Date of production		Medical device manufacturer
	Unique identification code of the device		Keep out of sunlight
	Protect from moisture		

	Cover: Wash at up to 30°C, do not bleach, dry flat in the shade, do not iron, do not dry-clean.
	Pillow core: Do not wash, do not bleach, dry flat, do not iron, do not dry-clean.

Product content:

Knitted polyamide/elastane (also fleece), viscose/polyester/elastane, polyester/cotton/elastane and polyester/elastane, white and ready-made dyed (acid, reactive and disperse dyes), 100% polyester (also coral fleece), white and dyed with disperse dyes, partly with white polyurethane foam, including accessories (sewing thread, zip, plastic snap/clasp, printed and woven label, elastic band); Pillow core made of thermoplastic polyurethane foam (memory foam).

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ACTION S.A.