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INSTRUCTIONS FOR USE

Issue number and date: 1 / 07.2025

Rehabilitation cushion for patient turning

Model PR100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to assist in the care of people with limited mobility, such as patients who are lying down, elderly or disabled. Its main function is to facilitate the safe and comfortable turning of the patient on the bed, allowing efficient repositioning of the body during nursing, rehabilitation, examination or daily care. The cushion stabilises the user's body, reduces the risk of injury and overload for the carer and supports pressure ulcer prevention. Thanks to its ergonomic shape, appropriately selected materials and the often present handles, it allows easy and secure grasping and moving of the patient, both at home and in medical facilities.

Intended use

Caring for patients with limited mobility in hospitals, care homes, rehabilitation centres and at home, enabling safe turning and positioning during care procedures. Use in rehabilitation, prevention of pressure sores and improvement of patient comfort during long-term immobilisation.

Contraindications

Unstable spinal fractures, acute femoral injuries requiring immobilisation and clinical conditions where the doctor will recommend avoiding repositioning due to the patient's condition. The need to consult a doctor before use to determine the feasibility of using a turning device for a particular clinical case.

Description of product construction

The rehabilitation patient turning cushion is characterised by a specially designed shape in the form of a half-shaft with an anatomical profile and universal openings for the lower limbs. The structure is made of highly flexible, warp-resistant foam that will not flex or collapse even under heavy loads. The product is covered with robust plastic upholstery. The cushion is equipped with side handles for easy turning and features careful stitching made using durable polyester thread. The whole is designed in a universal size for easy disinfection and hygiene.

First use

- Checking the integrity of the cushion and ensuring that it has no visible damage.
- Placing the pillow on the bed in a position that allows the patient to be positioned safely.
- Ensuring that the patient has no contraindications to a change in body position.
- Gently placing the patient on a pillow and making sure they are stable and safe.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

- Only use the cushion on stable and flat surfaces to prevent it from sliding or falling over.
- Before each use, make sure the cushion is not damaged and the turning mechanism is working properly.
- Always sit in the middle of the cushion to ensure even weight distribution and stability.
- Do not use the cushion when standing or when standing up without support - always move slowly and with control.
- Clean the cover regularly according to the manufacturer's recommendations to maintain the hygiene of the product.
- Do not use the cushion in a damp or wet environment to avoid slipping.
- Make sure the drain hose is not kinked or pinched to allow water to drain freely.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Regularly check the condition of the cushion before each use - it is not acceptable to use a damaged or soiled product.

- Ensure that the patient does not have contraindications to repositioning and monitoring their condition during turning.
- Ensure that the patient lies stable on the cushion and avoids sudden movements when turning to minimise the risk of injury.
- Exercise particular caution in patients with orthopaedic, neurological or elderly injuries.
- Store the cushion in a dry, clean place, away from sources of heat and direct sunlight.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Product content

- Core: 100% polyurethane (PU) foam with high elasticity and resistance to deformation.
- Cover: synthetic material, usually polyester or a polyester/cotton blend, easily removable and washable.
- Threads: durable polyester threads.
- No latex or metal components.

Technical data

- Packing size: 49 x 13 x 41cm

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Batch/series number
	Model		Warning
	Date of production		Medical device manufacturer
	Unique identification code of the device		Keep out of sunlight
	Protect from moisture		
		Pillow core: Do not wash, do not bleach, dry flat, do not iron, do not dry-clean.	

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ACTION S.A.