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Model NT200AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed for people who have difficulty sitting down or getting up from the toilet, such as seniors, people after injury, hip or knee surgery or with limited mobility. The toilet seat booster increases the height of the seat by 5, 10 or 15 cm (depending on the setting), making it easier to perform daily hygiene tasks and reducing the strain on joints. It is equipped with a lid and armrests that can be lifted to facilitate lateral transfer, for example from a wheelchair. The overlay is made of durable plastic, easy to clean and install (replaces a standard toilet seat), fits most standard toilets and provides stability for the user thanks to its robust construction with a maximum load capacity of 136 kg.

Intended use

The toilet elevation cap is designed for people with reduced mobility, such as the elderly, people after surgeries, or people with disabilities, making it easier for them to use the toilet safely and comfortably.

Contraindications

People with balance problems, coordination disorders, spinal instability or contraindications to sitting should avoid using the product due to the risk of falling and aggravating their condition.

Assembly

- Remove the standard toilet seat if necessary to create a flat mounting surface.
- Position the toilet seat booster centrally on the toilet bowl so that the two side knobs (visible from the front sides) are on the sides of the bowl.
- Tighten the two knobs by turning them clockwise until the toilet seat booster fits firmly against the shell and does not move.
- Check that the armrests are correctly attached and, if necessary, lift them up to facilitate lateral transfer.
- Ensure that the lid opens and closes freely and that the entire structure is stable before first use.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 136 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

1. Before each use, check that the booster is firmly attached to the toilet bowl and does not move when sitting down or standing up.
2. Armrests should be used for support when sitting down and standing up, but must not be stood on or overloaded to avoid structural damage.
3. Do not permanently mount the overlay - it is designed for easy installation and removal, which makes it easy to clean and allows regular inspection of the fixing.
4. After each use and on a regular basis, clean the pad with mild disinfectants, avoiding aggressive chemicals that may damage the surface.
5. Do not use the cap if it is damaged or if the fasteners are loose - repair or replace immediately if necessary.
6. Ensure that the user's feet are firmly on the floor when using the booster and that the hips are above knee level for safety and comfort.
7. The maximum load on the toilet seat booster should not be exceeded - follow the manufacturer's recommendations for user weight.
8. Do not leave children unsupervised when using the toilet seat booster - the product is designed for adults and people with limited dexterity.
9. Regularly check that all components (including fixing knobs and armrests) are in good working order and correctly fixed.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, petrol and all solvents, as these may cause damage to the product, visible defects and lead to a loss of warranty.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Product content

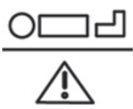
- Body and lid: plastic (polypropylene)
- Armrests: powder-coated metal (steel) with plastic pads
- Fastening elements (knobs, screws): plastic and corrosion-protected metal
- Seals/pads: flexible plastic (rubber)

Technical data

- Elevation height: 5 cm, 10 cm or 15 cm (adjustable in three levels)
- Seat width: 36 cm
- Width between armrests: 49 cm
- Depth: 48 cm
- Maximum load: 136 kg
- Material: plastic (PP - polypropylene), metal armrests with plastic pads
- Weight: approx. 4.5 kg

- Lid: yes, hinged
- Armrests: swivelling upwards
- Mounting: tool-free, with side knobs
- Fits most standard toilet bowls (does not fit toilets with concealed hinges)

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

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ACTION S.A.