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INSTRUCTIONS FOR USE

Issue number and date: 1 / 07.2025

Handrail support for getting up from the toilet

Model P100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed for the elderly, disabled or those with mobility difficulties, which significantly facilitates safe sitting down and getting up from the toilet. The design of the unit is based on a stable, free-standing frame with four legs fitted with non-slip feet, ensuring maximum safety during use. The support has comfortable armrests and is height-adjustable, so that it can be individually adapted to the user's needs and the height of the toilet bowl. The lightweight yet sturdy frame is made of powder-coated aluminium, ensuring durability and ease of maintenance. The device is designed for use both in the home and in care or rehabilitation facilities, providing the user with greater independence and comfort during daily toileting activities.

Intended use

A medical device designed to assist people with reduced mobility when sitting down and getting up from the toilet. The device provides additional support, stabilisation and safety for users with weakened muscular strength, balance problems or muscle-skeletal conditions, reducing the risk of falls and increasing independence during daily hygiene activities.

Contraindications

Significant imbalance or lack of trunk control that prevents independent and safe use. The use of the support is not recommended on slippery, wet or unstable surfaces or by people who are unable to maintain a sitting or standing position without additional assistance from medical personnel.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Sturdy, free-standing frame made of light metal (aluminium or powder-coated steel).
- Two side handrails ending in wide, comfortable plastic armrests.
- Four legs finished with non-slip feet for stability on the ground.
- Adjustable leg height to accommodate user height and toilet height.
- Two transverse reinforcements to stabilise the structure and increase the strength of the product.

Assembly

1. Remove all support components from the packaging and check for completeness.
2. Position the side arms with armrests on either side of the toilet bowl.
3. Connect the two side arms using the transverse stabilising tubes, inserting them into their respective holes and securing them with snaps.
4. Set the desired height of the support by adjusting the length of the legs and locking them with the buttons.
5. Make sure that all components are securely connected and that the non-slip feet are stuck to the floor.
6. Check the stability of the entire structure before first use.

Safety and usage rules

1. Before using it for the first time, make sure that the support has been properly assembled according to the manufacturer's instructions and that all components are securely connected.
2. Position the support on a level, dry and stable surface; avoid use on slippery, wet or unstable floors to minimise the risk of slipping or tipping over.
3. Check that the non-slip feet are firmly in place and not worn out; replace with new ones if necessary.
4. Before each use, check the stability of the structure and the tightness of all screws and connections.
5. Grip the handrails with both hands when sitting down and standing up to ensure maximum support and safety.
6. Do not load the support unevenly and do not use it as a ladder or bearer for activities other than those intended by the manufacturer.

7. Regularly clean the support with a damp cloth and mild detergent; do not use aggressive chemicals.
8. Do not use the support if you notice damage to the structure or looseness in the connections; consult service or the manufacturer if necessary.
9. The device is only intended for use by people who are able to maintain their own balance when sitting down and standing up; if there are significant imbalances or lack of trunk control, medical personnel should be consulted.

Maintenance

- Regular cleaning of the chair should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- Regularly clean all parts of the stand with a damp cloth and mild detergent; avoid using caustic agents and strong solvents.
- Thoroughly dry the support after each cleaning, paying particular attention to the metal parts to prevent corrosion.
- Systematically check the technical condition of the unit, including the stability of the connections, the tightness of the screws and the condition of the non-slip feet.
- If you notice play, structural damage or wear on components, replace them with new ones or contact the claims department.
- Store the support in a dry place, away from sources of moisture and direct sunlight.
- Do not make any modifications to the structure yourself that could affect the safe use of the product.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Frame made of powder-coated metal (steel / aluminium).
- Armrests made of plastic or covered with a soft pad.
- Legs finished with rubber, non-slip feet.
- Connecting and adjusting elements made of plastic.

Technical data

- Leg height adjustment: yes (in several stages)
- Height range: approx. 690 mm - 780 mm
- Overall width: approx. 540 mm
- Depth: approx. 470 mm
- Maximum load: 120 kg

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ACTION S.A.