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Laska inwalidzka

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Wstęp

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Models: Foldable; Adjustable

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A disabled cane is a medical device designed to assist the mobility of people with reduced mobility or after injury to the lower limbs. It provides stability and balance when walking on a variety of surfaces, both indoors and outdoors. The cane can be used to relieve pressure on the affected or weakened leg, and its correct adjustment to the user's height and regular inspection of its condition, including the rubber cap, are prerequisites for safe use. This product should be used as recommended by your doctor or physiotherapist.

Intended use

Disabled canes (crutches) are assistance equipment that provide stability and balance when walking on a variety of surfaces, both indoors and outdoors. They enable the body weight to be transferred from the legs to the upper body and can be used to relieve pressure on the affected or weakened leg. Crutches are particularly useful for people with limited mobility who cannot fully load their lower limbs due to injury, illness or permanent disability.

Contraindications

The disabled cane is not recommended for people with balance or coordination disorders, or for those who have upper limb injuries or dysfunctions that prevent them from supporting themselves properly. Inability to stand upright and severe perceptual impairment or intellectual disability requiring constant supervision are also contraindications.

User

The product can be operated independently by the user. A product designed for the layman.

Description of product construction

- Supporting handle - the part on which the hand rests to provide stability.
- Clamping ring - a component that connects the different parts of the cane, providing strength.
- Handle - a grip for holding the cane while walking.
- Height adjustment - a mechanism that allows the length of the cane to be adjusted to the user's height.
- The foot - the lower part of the cane that comes into contact with the ground - provides grip.
- Locking mechanism - an element that prevents the height adjustment from moving.

Description of use

1. Cane height adjustment:
 - Before first use, adjust the height of the cane using the adjustment mechanism.
 - The cane should be positioned so that its handle is at wrist level when you are standing upright with your arms freely lowered along your body.
 - Ensure that the locking mechanism is properly locked and the cane is stable.
2. Holding the cane correctly:
 - The cane should be held on the side of the healthy leg.
 - The hand should be slightly bent at the elbow when holding the handle.
 - Hold the handle securely, but do not press the cane too hard against the body.
3. Walking with a cane:
 - When walking, carry the cane and the affected leg forward at the same time.
 - Then transfer your weight to the cane and stand up with your healthy leg.
 - Repeat these steps to keep the movement smooth and stable.
4. Sitting down and standing up:
 - As you approach the chair, position yourself with your back to the seat.
 - Lean with your free hand on the railing or seat of a chair.
 - Slowly sit up, keeping the cane on the side of the healthy leg at all times.
 - When standing up, lean on your healthy leg and use a cane for support.
5. Folding and unfolding the cane:
 - In the case of a folding the cane, unfold it according to the manufacturer's instructions, making sure that all the components are correctly connected.
 - Once assembled, the cane should be stable and ready for use.
6. Inspection of technical condition:
 - Regularly check that all cane components are in good working order and securely fastened.
 - Pay particular attention to the rubber cap - it should be intact and provide good grip on the ground.
 - If the cap is worn or damaged, replace it with a new one.

7. Safety of use:

- Avoid walking on slippery or uneven surfaces unless necessary.
- Do not use the cane if you notice any damage to the structure or the adjustment mechanism.
- If in doubt about use, consult your doctor or physiotherapist.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 100 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

- Before each use, check that the cane is fully operational and that all components - in particular the height adjustment mechanism and the rubber cap - are properly attached and undamaged.
- Adjust the height of the cane so that it provides stable support and does not put undue strain on the upper limb.
- Always keep the cane on the side of the healthy leg and transfer your weight to the cane when stepping on the affected leg.
- Avoid walking on slippery, wet or uneven surfaces, which can increase the risk of slipping or falling.
- Do not use the cane if you notice any cracks, looseness or other damage to the structure.
- Regularly check the condition of the cane and replace worn or damaged components as necessary.
- Consult your doctor or physiotherapist if you have any doubts about correct use.
- The cane should only be used in accordance with its intended use and the recommendations of a specialist.

Maintenance and cleaning

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Maintaining the cleanliness of all components is key to maintaining the functionality of the product; they must be stored in appropriate conditions. Storage in rooms with high humidity (e.g. baths, laundries) or outdoors, combined with a lack of hygiene, can lead to a rapid loss of functional and aesthetic properties.
- Avoid exposing the cane to extreme weather conditions (rain, snow, intense sun), as some parts may change colour.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer

	Protect from moisture		Keep out of sunlight
---	-----------------------	--	----------------------

Material composition: aluminium (construction), TPR plastic (handle, adjustment elements), rubber (non-slip cap).

Colour: Various colour versions (see label).

Technical data:

- Height adjustment range: 80-90 cm or 79-102 cm (depending on the model)
- Height adjustment: 2.5 cm increments
- Weight: 0.35–0.4 kg

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ACTION S.A.