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Model KP600AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to be mounted on the wall in a shower cubicle, designed to provide comfort and safety for the elderly, disabled people or those with mobility difficulties. Its foldable design saves space when not in use and its stable and robust construction allows safe use while bathing. The seat is made of water-resistant, easy-to-clean materials, and the ergonomic shape and non-slip surface further enhance user comfort and safety. This chair provides a practical and aesthetically pleasing solution that promotes independence during daily hygiene, minimising the risk of falling in the shower.

Intended use

The medical device is intended to facilitate the performance of hygiene-related at home and in hospital, especially for people with reduced mobility. It can be used both in the shower and in the bath, providing stability and comfort while bathing.

Contraindications

People with balance problems, coordination disorders, spinal instability or contraindications to sitting should avoid using the product due to the risk of falling and aggravating their condition.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- A folding design that allows the chair to be easily folded against the wall to save space in the shower cubicle when not in use.
- The seat is made of durable PVC and measures 40 x 32 cm for comfort and water resistance.
- Adjustable seat height from 42 to 52 cm, allowing the chair to be adapted to the individual user's needs.
- Robust construction with high-quality materials, guaranteeing durability and stability even in high humidity conditions.
- Rubber caps on the legs to prevent the chair from slipping and to protect the bathroom surface from damage.
- With an aesthetically pleasing design and careful finish, the chair blends in with the design of a modern bathroom.
- Wall-mounted for permanent attachment to the wall in the shower area for maximum user comfort and safety.

Assembly

- Site selection: The chair should be mounted on the wall in the shower cubicle, at a suitable height for comfort and easy access.
- Surface preparation: The wall should be even, solid and made of a material that allows secure fixing (e.g. concrete, brick, solid blocks); installation on thin partitions or plasterboard without additional reinforcement is not recommended.
- Marking of mounting points: Using a spirit level, determine and mark the hole locations according to the spacing of the holes in the chair handles.
- Drilling of holes: Holes should be drilled at the designated locations with the appropriate diameter drill to suit the wall plugs.
- Handle attachment: Insert the wall plugs into the prepared holes and then screw the chair mounting brackets to the wall using screws.
- Suspension and seat protection: Once the brackets have been attached, hang the seat on the structure and check that the assembly is correct, making sure that the chair is stable and secure.
- Load test: Before first use, it is advisable to test the chair for stability and maximum load, according to the manufacturer's instructions.

Safety and usage rules

- Make sure the chair is securely mounted on a flat, solid wall and check that the fixings are tightened and show no signs of damage before each use.
- Only use the chair for its intended purpose - only to sit while bathing; do not stand, kneel, use as a ladder or to store objects.
- Observe the maximum permissible load specified by the manufacturer to avoid damage to the product and the risk of falling.
- Regularly inspect the technical condition: check bolts, connections and seat surface; if play or damage is detected, stop using immediately until repaired.
- Avoid contact with sharp objects and do not sit on the chair in wet clothes, which can dye or damage the seat material.
- Once the chair is folded, make sure it is secured to prevent accidental unfolding or falling.
- Do not assemble or use the chair near heat sources (e.g. radiators, cookers) to prevent deformation or damage to materials.
- Take special care in the presence of children, making sure that the product meets the relevant safety standards and does not pose a risk of pinching fingers or falling over.
- When cleaning the chair, follow the manufacturer's recommendations and regularly check the condition of the anti-corrosion coating of the metal components.
- Do not use the chair if the surface underneath it is slippery or wet to avoid the risk of slipping and falling.

Maintenance

- Regular cleaning of the chair should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- After each use, rinse the seat and chair structure with water to remove soap residues and dirt, which prevents deposits and the growth of mould and fungi.
- Regularly clean the product with a mild detergent and a soft sponge or cloth, avoiding aggressive chemicals and rough tools that may damage the surface of the seat and structural components.
- Thoroughly dry all components after cleaning, especially at joints and fastenings, to reduce the risk of corrosion and bacterial growth.
- Periodically check the condition of bolts, fixings and hinges, tightening them as necessary and replacing damaged or worn parts immediately.
- Check the condition of the anti-corrosion coating of metal components and, if corrosion centres appear, remove them gently and then protect them with a suitable preparation.
- Do not leave the chair submerged in water for long periods of time and avoid prolonged contact with highly humid environments unless necessary.
- Store the chair in a dry place if it is not used for a long time to prolong its life and maintain its aesthetic appearance.

Storage and transport conditions

The chair should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

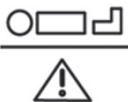
Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Seat: durable PVC (polyvinyl chloride) plastic, water-resistant and easy to clean.
- Under structure: high-quality metal, aluminium / powder-coated steel.
- Rubber caps on the feet: non-slip elements to protect the bathroom surface from scratches and provide stability during use.

Technical data

- Seat dimensions: 32 x 40 cm.
- Seat height adjustable from 42 to 52 cm.
- Body mass: 2.0 kg.
- Maximum load: 120 kg.
- Foldable: yes, the design allows the chair to be folded against the wall.
- Rubber caps on the feet: prevent slipping and protect the bathroom surface.
- Mounting: wall-mounted, for permanent installation in the shower enclosure.

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ACTION S.A.