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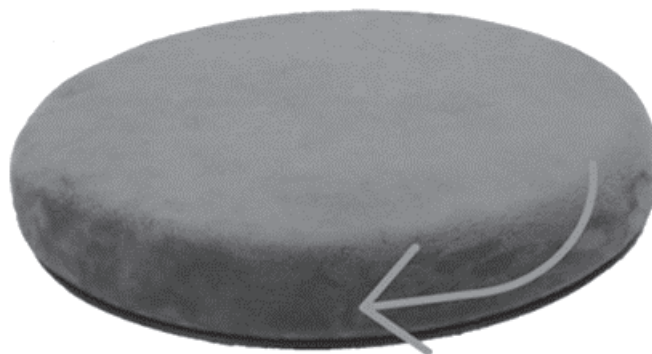
Model: PO600AM

Wstęp

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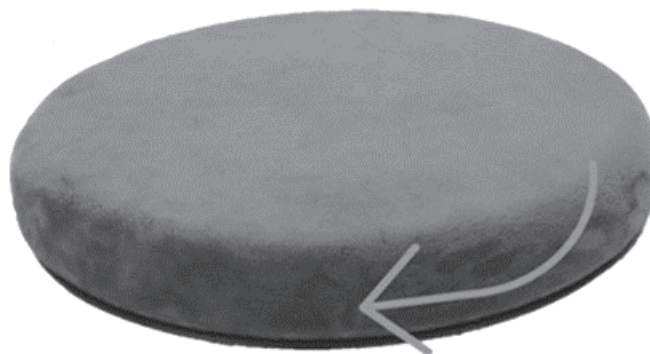
Model PO600AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to facilitate repositioning while sitting, especially for people with reduced mobility, seniors and patients after injury or surgery. The functionality of the cushion is to allow a smooth 360-degree rotation without excessive effort, making it significantly easier to get up from a car, chair or armchair and to get in and out of vehicles. The cushion increases the comfort of the user, reduces the risk of muscle and joint strain and supports the daily functioning of people who have difficulty changing their sitting position independently.

Intended use

A product designed for rehabilitation and support of patients with limited mobility, including those following spinal and hip injuries, orthopaedic surgery and seniors with mobility problems. The medical device is used in physiotherapy as an aid to rotational exercises, as well as in everyday use to facilitate transfer from a sitting position, reducing joint strain and the risk of falls when getting up from a chair, armchair or car.

Contraindications

Acute arthritis, significant hip or spinal instability and fresh injuries in the pelvic region. In addition, it is contraindicated in people with advanced osteoporosis, an unhealed fracture of the pelvic bone or spine and severe imbalances.

Description of product construction

It consists of a round, soft seat made of memory foam, which provides comfort and an optimal fit to the user's body. The cushion is equipped with a mechanism that allows a free 360-degree rotation, which makes it easy to change position while sitting, and the underside of the cushion has a non-slip surface to prevent the product from sliding on the substrate. The whole is covered with a pleasant-to-touch fabric that can be removed for washing, making it easy to maintain hygiene.

First use

- Remove the cushion from its packaging and remove the protective materials.
- Check the cushion for visible damage.
- Place the cushion on a stable seat, non-slip side down.
- Sit slowly in the middle of the cushion.
- Gently make a body movement to check the freedom of rotation.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

- Only use the cushion on stable and flat surfaces to prevent it from sliding or falling over.
- Before each use, make sure the cushion is not damaged and the swivel mechanism is working properly.
- Always sit in the middle of the cushion to ensure even weight distribution and stability.
- Do not use the cushion when standing or when standing up without support - always move slowly and with control.
- Clean the cover regularly according to the manufacturer's recommendations to maintain the hygiene of the product.
- Do not use the cushion in a damp or wet environment to avoid slipping.
- Ensure that the drain hose is not kinked or pinched to allow water to drain freely.

Maintenance

- Regularly clean the cushion cover in accordance, usually by removing it and washing it by hand or in the washing machine at a low temperature.
- Avoid using aggressive detergents and bleach, which can damage the material.
- The swivel mechanism can be wiped with a dry or slightly damp cloth; do not immerse the cushion in water.
- Make sure all components are completely dry before using them again.

- Store the cushion in a dry place, away from heat sources and direct sunlight to prevent deformation and prolong the life of the product.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Product content

Knitted polyamide/elastane (also fleece), viscose/polyester/elastane, polyester/cotton/elastane and polyester/elastane, white and ready-made dyed (acid, reactive and disperse dyes), 100% polyester (also coral fleece), white and dyed with disperse dyes, partly with white polyurethane foam, including accessories (sewing thread, zip, plastic snap/clasp, printed and woven label, elastic band); Pillow core made of thermoplastic polyurethane foam (memory foam). The rotary base is made of robust plastic, polypropylene (PP) with an anti-slip material (rubber).

Technical data

- Diameter 39 cm
- Height: 5 cm
- Maximum permissible weight: 120 kg

Explanation of symbols

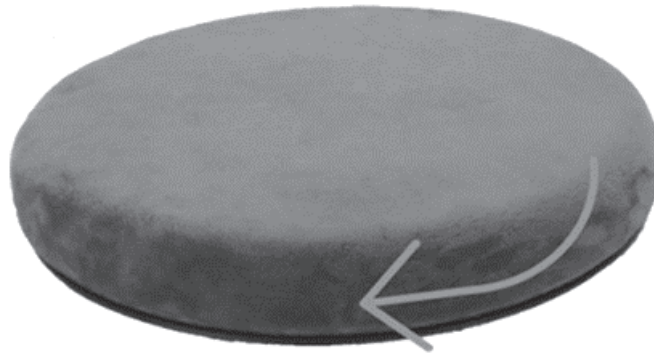
	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum permissible weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight
	Coating: Wash at up to 30°C, do not bleach, dry flat in the shade, do not iron, do not dry-clean.		
	Pillow core: Do not wash, do not bleach, dry flat, do not iron, do not dry-clean.		

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ACTION S.A.