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INSTRUCTIONS FOR USE

Issue number and date: 1 / 07.2025

Inflatable washing tub for handicapped persons

Model DW100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed for people with limited mobility, chronically ill patients, stroke patients and seniors that allows bathing to be carried out directly on the bed without having to move the patient to the bathroom. Made of durable PVC 0.3-0.5 mm thick, they have an ergonomic shape that adapts to the body, ensuring comfortable use while sitting and lying down. They are widely used in care homes, hospitals, hospices and during home care, significantly facilitating hygiene and reducing the risk of injury associated with patient handling.

Intended use

The product is intended for use in hospitals, nursing homes, hospices and during home care for patients with limited mobility, chronically ill patients, stroke patients, people with spinal injuries and seniors who cannot be safely transferred to a traditional bathroom. The product enables safe bathing to be carried out directly on the hospital or home bed, significantly reducing the risk of patient handling related injuries and ensuring that the appropriate level of hygiene is maintained for not self-reliant persons.

Contraindications

Open wounds, fresh surgical scars, inflammation of the skin and bacterial or fungal infections that may be exacerbated by contact with water and moisture. In addition, the product should not be used for patients with severe circulatory disorders, respiratory insufficiency requiring constant medical supervision and those with a body weight in excess of the maximum load of the bathtub, and where it is not possible to ensure an adequate water temperature of no more than 40°C.

Description of product construction

The inflatable wash tub for handicapped persons is constructed of durable PVC material and consists of several main components: an air chamber (walls and bottom), an inflate/deflate valve, a drain valve on the bottom of the tub (Drain), as well as a set of accessories such as a hand pump and a repair kit. Measuring 1900 × 900 × 200 mm, the tub is designed to be quickly inflated and deflated, allowing for easy handling and storage once deflated. The entire unit has been designed for comfort, safety and ease of hygiene, and the ergonomic raised walls prevent water spilling during bathing.

Installation / first use

1. Roll out the tub on a flat, level surface, free of sharp objects.
2. Using a pump (hand or electric), inflate the individual air chambers of the tub, starting with the largest; once each chamber is inflated, quickly close the valve to prevent air escaping.
3. Make sure the tub is fully inflated and stable and all valves are tightly closed.
4. Connect the drain hose to the appropriate valve, adjusting the length as required.
5. Before first use, rinse the inside of the tub with water and a mild detergent, then rinse and dry thoroughly.
6. Before bathing the patient, check the temperature of the water (it should not exceed 40°C) and make sure the bath is correctly set up and safe to use.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 130 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

- Control the pressure when pumping to prevent rupture of the tub.
- Place the bathtub on a flat, dry, clean surface free of sharp objects; do not use it near a heat source or fire.
- Once inflated, only use by one person at a time.
- Before filling with water, first pour in cold, then warm water; the water temperature should not exceed 40°C.

- After bathing, empty the tub through the bottom drain and dry it thoroughly.
- Do not use sharp tools, caustic cleaners or organic substances (e.g. solvents) to clean the tub.
- Disinfect and dry the tub after use and store it in a dry, ventilated place, away from corrosive substances.
- Do not use the product if it is visibly damaged or leaking, and do not exceed the maximum recommended load.
- Take care during transport and storage not to puncture or damage the material.

Maintenance

1. Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
2. Avoid the use of: strong detergents, abrasives, bleach, petrol and all solvents, as these may cause damage to the product, visible defects and lead to a loss of warranty.
3. After each use, the device should be disinfected using disinfectants designed for disinfection of surfaces in contact with the skin, according to the manufacturer's recommendations, and then thoroughly rinsed and dried to prevent the growth of bacteria or fungi.
4. Store the product in a dry, well-ventilated place, away from heat sources and corrosive substances; do not fold or store the product in a damp state to avoid mould formation and material damage.
5. Regularly check the condition of the product, paying attention to the tightness of the air chambers and the absence of mechanical damage; if minor leaks are detected, the repair kit supplied with the product can be used.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.

- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

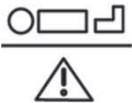
Product content

The product is made of high-quality, durable PVC (polyvinyl chloride), with a thickness typically in the range 0.3-0.5 mm, which ensures flexibility, resistance to mechanical damage and ease of hygiene.

Technical data

- Dimensions: 1900 × 900 × 200 mm
- Maximum permissible user weight: 130 kg
- Weight: approx. 2.5 kg
- Set includes: inflatable tub, hand pump, repair kit, drain hose
- Valve system: inflation/deflation valve, drain valve

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum permissible weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

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ACTION S.A.