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INSTRUCTIONS FOR USE

Issue number and date: 1 / 07.2025

Rehabilitation rotor with counter

Model RR100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

Medical device intended for rehabilitation exercises of the upper and lower limbs in the home and in medical facilities. The device consists of a powder-coated steel frame with folding capability, a rotary motion system with two pedals fitted with rubber non-slip bands, an adjustable resistance mechanism and a digital counter powered by a 1.5V AG13 battery. Functionality includes a display of the number of calories burned, exercise duration, number of revolutions per minute and the total of all revolutions performed, with automatic start, stop and pause options (functionality is illustrative).

Intended use

The device is designed for patients with muscular atrophy, contractures caused by long-term immobilisation, people after injuries, strokes and those requiring the restoration of motor function and re-learning of the correct movement pattern after damage to the neurological system. The rotor can be used for both leg exercises (placed on the floor while sitting) and arm exercises (placed on a table), making it a versatile rehabilitation tool to improve blood circulation, prevent blood clots and strengthen muscles.

Contraindications

The rehabilitation rotor is not recommended for patients with acute vascular conditions such as deep vein thrombosis, a tendency to bleeding, during pregnancy and in areas with damaged skin or an active disease process (infection, cancer, post-radiotherapy condition). Particular attention should be paid to patients with sensory impairments, communication problems, skin diseases or regenerating nerves, in whom the use of the device requires individual assessment and supervision by a specialist.

Preparation for installation

- Check all rotor components for completeness before starting assembly
- Make sure all bolts and nuts are accessible

Order of installation:

- Unfolding the base - unfold the lower parts of the rotor legs by removing them from the folded position
- Leg locking - push in the locking pin sequentially on either side of the frame to lock the legs in the extended position
- Mounting the top - use M*40 MM bolts and nuts to secure the top of the rotor to the base
- Installation of the adjustment knob - screw on the triangular knob at the top of the rotor, which is used to adjust the resistance
- Pedal assembly - attach pedals with non-slip rubber bands to the rotor arms
- Meter installation - mount the digital meter powered by a 1.5V AG13 battery in the designated location
- Stability check - ensure all anti-slip pads are correctly positioned on the base

Final inspection:

- Check that all screws are tightened
- Test the operation of the resistance control
- Ensure that the rotor stands firmly on a flat surface

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

- Do not stand on the pedals - the machine is not designed to support body weight.
- Proper position during exercise - always sit on a chair, sofa or bed when using the rotor.
- Pre-use inspection - check that all screws, bolts, knobs, frames and brackets are correctly fitted before each use.
- Supervision of persons requiring assistance - patients, disabled and elderly persons must use the device when accompanied by another person.
- Caution when transporting - pay particular attention to safety when moving the unit.

Preparation for exercises

- Warm-up - perform appropriate stretching exercises before use to prevent injury.
- Fit the bands - place your feet on the pedals and adjust the circumference of the band to the circumference of your foot, which will prevent slipping during exercise.
- Medical consultation - consult your doctor before starting exercise, especially if you have health problems.

Display operation

To activate the display, press the red button once or start pedalling. The display automatically switches off after 4 minutes of inactivity. All display functions activate when you start pedalling and stop when you stop. To switch between functions, press the red button until the arrow points to the desired function.

Display functions:

- **CNT:** Number of rotations performed
- **TIME:** Duration of training
- **RPM:** Revolutions per minute
- **CAL:** Estimated number of calories burned
- **SCAN:** Switching between functions (TIME > CNT > RPM > CAL)

Battery replacement:

Remove the display unit from the socket. On the back of the display unit, lift the battery latches and slide out the battery. Insert a new 1.5V AG13 battery with the positive pole down and close the latches. Insert the display unit back into the socket.

Inspection:

- Regular checking - check and tighten all parts of the unit every three months.
- Original spare parts - use only original manufacturer's spare parts.
- Respond to worrying symptoms - consult a healthcare professional if you experience pain, allergic reactions or other worrying symptoms.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Check and tighten all bolts, nuts and fixings regularly, every three months, to ensure safe use and stability of the unit.
- Store the rotor in a dry place, away from moisture, direct sunlight and heat sources to prevent corrosion and deformation of the components.
- Do not use the appliance if there is visible damage; if necessary, replace worn parts only with original manufacturer's components.
- Disconnect the unit from the power supply (if electric) before carrying out any maintenance.
- Do not store the rotor outdoors or in rooms with high humidity (e.g. bathrooms, laundries) to avoid rapid wear and loss of performance.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

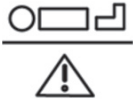
Product content

- The frame and base are made of powder-coated steel.
- The pedal arms and components are made of chrome-plated steel tubes.
- Pedals are made of durable plastic, usually ABS / PP.
- The foot bands are made of flexible plastic.
- The resistance adjustment knob is made of plastic.
- The electronic counter has an LCD display and a plastic housing.
- The anti-slip pads on the feet are made of rubber or other elastomer material.

Technical data

- Dimensions: 50 x 36 x 26 cm (length x width x height)
- Weight: approx. 2.3 kg
- Counter: electronic, with LCD display, battery-operated (1.5V AG13)
- Adjustment of resistance: stepless, with dial

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum permissible user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

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ACTION S.A.