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Stopień schodek do wanny, PREMIUM PLUS

Model: SW200AM

Wstęp

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Model SW200AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to facilitate safe entry and exit of the bathtub for people with reduced mobility, including seniors, people after injury, surgery or with balance disorders. Made from durable and moisture-resistant materials, this step stool is equipped with non-slip pads and a stable surface to minimise the risk of slipping and falling when using the bath. Thanks to its compact dimensions and lightweight design, the step stool is easy to carry and store, and the adjustable height allows it to be adapted to the individual user's needs. This product increases independence and comfort during daily hygiene activities, working well in both home and care settings.

Intended use

A medical device designed to facilitate safe entry into and exit from the bathtub for people with reduced mobility, providing stable support and minimising the risk of slipping during daily hygiene activities.

Contraindications

Do not use with imbalance, coordination disorders, weight over 120 kg, step damage or slippery ground.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Metal frame - the step structure is based on solid, chrome-plated tubes for strength and stability.
- Step surface covered with an anti-slip mat - the top of the step has a black grooved mat to prevent feet slipping during use.
- Four widely spaced legs - ensure even weight distribution and increase the stability of the entire structure.
- Legs finished with rubber pads - each leg is fitted with non-slip rubber feet to protect the floor from scratches and prevent the step from sliding.
- No sharp edges - all components have rounded shapes, making them safer to use.
- Compact dimensions - the step stool is small and lightweight, making it easy to carry and store.

Assembly

- Remove the stepladder from the packaging and check that all components are complete and undamaged.
- Place the step stool on a flat, stable surface where it will be used, preferably right next to the bath.
- Check that all the step legs are evenly aligned and that the rubber pads are firmly in place to prevent the step from moving during use.
- No screwing or additional assembly is required - the step is ready for use as soon as it is taken out of the packaging.
- Before first use, it is advisable to wipe the anti-slip surface with a damp cloth to remove any dust or production residue.

Safety and usage rules

- Before each use, check that the step stool is stable and that all legs are correctly attached and positioned on a level, dry surface.
- Make sure the rubber caps on the legs fit snugly and are not damaged.
- Do not exceed the maximum permissible load.
- Only use the step stool for its intended purpose - as an aid to getting in and out of the bath.
- Do not use the step stool if you have a severe imbalance or coordination disorder without third-party supervision.
- Do not use the step stool if it is damaged or visibly cracked.
- After using the step in a damp environment, it is recommended to dry it and clean it regularly to prevent slipping and to maintain hygiene.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Wipe the step stool down dry after each use to prevent moisture build-up and deposits.
- Regularly check the condition of the rubber leg caps and adjustment elements - replace them with new ones if worn or damaged.
- Avoid prolonged contact of the stepladder with water - do not leave it in the bath or on a wet surface for long periods of time.
- Store the step stool in a dry place, away from direct sunlight and heat sources.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

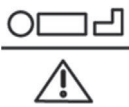
Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Frame: chrome-plated steel or stainless steel (corrosion- and moisture-resistant metal).
- Step surface: plastic with non-slip texture (rubber or PVC).
- Leg pads: non-slip rubber.

Technical data

- Width of step surface: 41 cm
- Depth of step surface: 22 cm
- Step height: 23 cm
- Overall width (from foot to foot): 35.5 cm
- Overall depth (from foot to foot): 25.5 cm
- Frame material: chromium-plated steel
- Step surface: non-slip plastic lining
- Legs: finished with anti-slip rubber pads
- Maximum load: 120 kg
- Weight: approx. 2 kg

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ACTION S.A.