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INSTRUCTIONS FOR USE

Issue number and date: 1 / 07.2025

Bathtub bench, 2-in-1 with rotating disc

Model LW400AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to facilitate safe entry and exit of the bathtub and hygiene activities for people with reduced mobility, seniors or patients undergoing rehabilitation. Featuring a stable structure and non-slip features, the bench ensures comfort and safety in use, while the rotating disc allows for easy turning when sitting down and standing up, significantly reducing the risk of falling and facilitating independence during bathing.

Intended use

The medical device is designed to assist people with reduced mobility, the elderly, patients after surgery, with neurological diseases or balance problems, enabling them to safely and comfortably perform everyday hygiene activities, such as bathing or showering, both at home and in healthcare facilities.

Contraindications

Any patient condition that prevents the patient from safely adopting a sitting position during bathing, as well as disorders of perception or consciousness that may impede the correct and safe use of the device.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Seat: Rectangular, made of durable, waterproof plastic; fitted with perforations (drain holes) to allow water to drain freely.
- Side handles: Four curved aluminium arms with non-slip rubber pads on the ends, ensuring the bench remains stable on the bathtub rim and protects the bathtub from scratches.
- Width adjustment: The design allows the bench to fit a variety of bath widths, increasing the versatility of use.
- Lightweight design: Made of lightweight, corrosion-resistant materials for easy handling and installation.
- Ease of cleaning: The smooth surfaces and materials used allow the bench to be kept clean quickly and easily.
- No backrest: The model has no backrest, allowing easy access from both sides and comfortable sitting and standing up.
- Intended use: For use in the bath by people with limited mobility, seniors and people following injury or surgery.

Assembly

- Choosing a bathtub: Make sure the bathtub has the right width and shape to fit the bench; this model has an adjustable width to suit the dimensions of the bathtub.
- Width adjustment: Extend or retract the aluminium side arms to fit the bench firmly on the edges of the bath.
- Bench placement: Once the width is set, place the bench on the bath so that the side handles rest stably on the edges of the bath.
- Handle protection: Check that the rubber caps on the ends of the handles fit snugly into the bathtub to prevent slipping and scratching.
- Assembling the rotating disc: Ensure that the blue rotating disc is correctly fixed and spins freely, allowing the user to rotate comfortably.
- Stability test: Before using it for the first time, check that the bench is stable, does not wobble and holds well on the bathtub.
- Prescriptions for use: Once assembled, the bench is ready for use, allowing you to sit, turn and stand up safely while bathing.

Safety and usage rules

1. Before each use, make sure that the bench is correctly installed, stable and that all its fixings are well aligned with the edge of the bathtub.
2. Check that the rubber pads are in full contact with the surface of the bathtub - this prevents the bench from sliding and protects the bath from scratching.

3. Before sitting on the bench, make sure the seat is dry and clean and the drain holes are not blocked.
4. Be careful when getting on and off the bench - first sit on the edge of the seat and then slowly move your legs inside the bath, holding onto the handrails or bathroom handles if available.
5. Do not exceed the maximum load allowed on the bench.
6. Do not use the bench if you have imbalances, lack of trunk stability or contraindications to sitting.
7. Regularly inspect the condition of the bench - check for damage, loose fixings and that the seat surface is not slippery or cracked.
8. It is advisable to use additional non-slip accessories such as bathroom mats and to keep a phone or emergency bell nearby in case you need to call for help.
9. Always use the bench in accordance with the manufacturer's instructions and do not modify the structure yourself.
10. After the bath, clean and dry the bench thoroughly to maintain its hygiene and durability.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Wipe the bench dry after each use to prevent moisture and deposits from building up on the seat surface and metal parts.
- Regularly wash the bench with a mild soapy water solution or gentle detergent; avoid abrasive cleaners, bleach and aggressive chemicals that can damage the surface or cause corrosion.
- Check drain holes - clean out sediment if necessary to ensure free drainage and hygienic use.
- Check the condition of the rubber pads on the handles - replace them if worn or damaged to maintain the stability and safety of the bench.
- Periodically inspect the entire structure for play, cracks or other damage; if abnormalities are found, discontinue use until the component is repaired or replaced.
- Store the bench in a dry place after use, especially if it will not be used for a long time.
- Do not carry out modifications or repairs yourself that could affect the safety or durability of the product.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

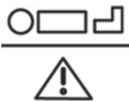
Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Seat: Durable, water-resistant plastic (polypropylene)
- Rotating disk: Plastic with non-slip surface (TPR)
- Side handle: Reinforced plastic (polypropylene) with anti-slip coating
- Legs and support elements: Plastic with non-slip rubber tips (rubber or TPR - thermoplastic rubber)
- Metal components (if any): Aluminium or stainless steel (for possible structural reinforcement)

Technical data

- Overall length of bench: approx. 70-75 cm
- Overall bench width: approx. 30-35 cm
- Rotating disc diameter: approx. 30 cm
- Height (thickness) of bench: approx. 5-7 cm (without handle)
- Handle height: approx. 10-12 cm above seat surface
- Maximum load: 120 kg
- Width adjustment: can be adapted to the width of the bathtub (usually in the range of 38-62 cm)
- Product weight: approx. 2.5 kg

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ACTION S.A.