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Ławka wannowa, z oparciem

Model: LW100AM

Wstęp

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Model LW100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to facilitate daily hygiene activities for people with reduced mobility, seniors and those following injury or surgery. It is an supportive device that allows people to safely and comfortably use the bathtub or shower without having to stand, significantly minimising the risk of falls and slips. The bathtub bench with backrest enables the user to assume a seated position while bathing, which increases the user's independence and comfort in performing hygiene tasks, while the backrest provides additional back stabilisation. The equipment is lightweight, easy to carry and assemble, with a non-slip seat surface and stable feet to minimise the risk of slipping and guarantee safety during use. It is designed to come into contact with water and moisture, so it works well in both home and healthcare settings to support people with limited mobility when bathing or showering.

Intended use

The medical device is designed to assist people with reduced mobility, the elderly, patients after surgery, with neurological diseases or balance problems, enabling them to safely

and comfortably perform everyday hygiene activities, such as bathing or showering, both at home and in healthcare facilities.

Contraindications

Any patient condition that prevents the patient from safely adopting a sitting position during bath, as well as disorders of perception or consciousness that may impede the correct and safe use of the device.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Aluminium frame - lightweight, corrosion and moisture resistant, provides stability to the entire structure; often allows width adjustment to fit the bench to different baths.
- Plastic (PVC) seat - sturdy, easy to clean, water-resistant; the surface can be perforated or non-slip, increasing user safety.
- Plastic (PVC) backrest - provides back support, increases comfort and safety during bath.
- Feet finished with rubber caps - non-slip, prevents the bench from sliding on the bathtub surface and protects against scratching the surface.
- Side handrails - steel, make it easier for the person using the bench to stand up and sit down, increasing their independence and safety.

Assembly

1. Remove all bench components from the packaging and check their completeness and condition.
2. Mount the legs to the seat frame by inserting them into the appropriate holes and secure with the locking mechanism or screws, if provided.
3. Fix the backrest by sliding it into the dedicated slots in the back of the seat and secure it according to the manufacturer's instructions.
4. Set the height of each leg, adjusting the length by extending or retracting the tube and locking it in the desired position.
5. Check that the rubber caps on the ends of the legs fit snugly to prevent the bench from moving.
6. Once the installation is complete, test the stability of the bench by gently pressing it down and checking that it does not wobble on the ground.
7. Height adjustment: Adjust the height of the legs to suit you, making sure the locks are fastened correctly and all legs are the same length.

Safety and usage rules

1. Before each use, check that the bench is correctly assembled and stably positioned in the bathtub or shower.
2. Make sure that all the rubber caps on the legs are firmly in place to prevent the bench from moving.
3. Check the condition of the bench and its cleanliness, especially of the seat and back surfaces.
4. Adjust the height of the legs to the level of the bathtub or floor to ensure user comfort and safety.
5. Sit down slowly, holding onto the handrails or stable parts of the bench, do not make sudden movements.
6. When bathing, do not lean violently or shift your body weight to one side to avoid tipping the bench over.
7. Only use the bench for its intended purpose and only by people who can balance themselves in a sitting position.
8. Do not use the bench if any component is damaged or if structural instability occurs.
9. It is recommended that initial use and installation be carried out in accordance with the manufacturer's instructions and under the supervision of a supervisor if the user requires support.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Wipe the bench dry after each use to prevent moisture and deposits from building up on the seat surface and metal parts.
- Regularly wash the bench with a mild soapy water solution or a mild detergent; avoid using abrasives, bleach and aggressive chemicals that can damage the surface or cause corrosion.
- Check drain holes - clean out sediment if necessary to ensure free drainage and hygienic use.
- Check the condition of the rubber pads on the handles - replace them if worn or damaged to maintain the stability and safety of the bench.
- Periodically inspect the entire structure for play, cracks or other damage; if abnormalities are found, discontinue use until the component is repaired or replaced.
- Store the bench in a dry place after use, especially if it will not be used for a long time.
- Do not carry out modifications or repairs yourself that could affect the safety or durability of the product.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

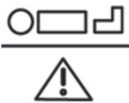
Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Aluminium - used for the frame and legs of the bench; this material provides lightness, corrosion resistance and structural strength.
- Plastic (PVC) - the seat and backrest are made of this material; the plastic is water-resistant, easy to clean and safe in contact with the skin.
- Rubber / thermoplastic - used in the caps at the ends of the legs, which provides anti-slip properties and protects the surface of the bathtub from scratching.

Technical data

- Seat width: approx. 70 cm
- Seat depth: approx. 40 cm
- Seat height (adjustable): approx. 40-55 cm
- Overall height with backrest: approx. 75-85 cm
- Maximum load: 120 kg
- Weight of bench: approx. 4 kg

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ACTION S.A.