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USER MANUAL
BATHLIFT
REF: 01.01.BATHLIFT

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l..
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l. declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO S.r.l. in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l. declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO S.r.l. comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO S.r.l. guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

The electric lift has been designed to facilitate the access and exit of a person in a bathtub.

WARNING!

- Do not use the lift as an aid to get in or out of other rooms except the bathtub.
- Do not use as a support, as a ladder, or transport platform or for other intentions of similar magnitude.
- Other uses outside of the one allowed may cause serious injury.

DESCRIPTION

The lift has a basic structure that allows the lifting of the seat from a minimum of 8 cm to a maximum of 48 cm. The actuator is located on the back of the backrest. The lifters can be raised, lowered and the slope of the backrest adjusted. The actuator movements can be controlled from the remote control, connected to the backrest. The battery has the energy necessary for the actuator to move. The device lowers only if there is enough energy to get up again, this is due to its integrated energy control.

WARNINGS

- Always pay attention to the presence of children.
- People with severe physical or mental disabilities should never be left alone in the bathroom.
- The seat may slip if the non-slip surface do not stick properly. Clean the surface before use.
- DO NOT move the lift once it is placed.
- Replace the non-slip surface if it is damaged or worn.
- Always sit in the center of the lift and keep the weight at the center during the washing movements.
- When lifting or lowering the lift, do not touch the edges of the bathtub, do not leave your hands under the seat, this can cause injuries.
- Use the supplied battery charger.
- The battery charger must be used outside the bathroom, in a dry environment.
- Do not connect the battery charger to power sources until the battery is connected to it.
- Do not place the battery near hot, flammable objects.
- Always disconnect the battery before transporting the lift.
- The batteries are rechargeable and replaceable, but can only be replaced by qualified persons. Do not open the batteries. Discard used batteries in the appropriate container.
- Repairs must be carried out by authorized personnel.
- The actuator is not designed for use below water level. To avoid dangers and damage do not use the lift while the actuator is immersed in water.
- Do not use a damaged lift.
- Do not overload the person lift to avoid serious injury.
- Recharge the battery before storing the lift for extended periods of time.
- Do not exceed the maximum capacity of 140 Kg.

BEFORE EVERY USE

- Check the correct assembly of the device.
- Always check the state of the mechanical parts.
- Check the condition of the non-slip surface and their adherence to the bathtub.

USE

WARNING!

- The battery must be recharged for at least 8 hours before first use.
- It is recommended to recharge it after each use.

How to recharge the battery

WARNING!

Always recharge using a socket outside the bathroom, in a dry place.

Follow the procedures below to recharge the batteries:

1. Remove the batteries from the lift by releasing the locking tab on the back of the backrest.

Posterior view of the lift

Battery cable to attach at the battery charger

2. Connect the battery to the supplied battery charger.
3. Insert the plug of the battery charger into the mains socket.

IMPORTANT!

While using the lift, a light will be visible on the remote control.

Green / blue light: the battery is charged.

Red light: the battery is almost empty.

Battery connected to the battery charger

How to install the bathtub lift

Place the person lift in the horizontal part of the bathtub.

Fix the device by pressing it down.

Before each use, make sure that:

- a. The surface is clean. Check the effective adhesion of the lift. It not adhere to textile or non-slip surfaces. Avoid these types of surfaces.
- b. Leave enough space to allow the backrest to move. Make sure that there are no obstacles that prevent the movement of the lift, such as taps.

WARNING!

- The battery must be fully charged before use.
- Before using the lift consider that you may need help.
- Get advice from qualified personnel such as a therapist in case of doubts about the correct use of the lift.

1. Raise the lift until you reach the bathtub side with the seat.
2. Facing outwards, sit on the lifts and then move around the seat trying to position yourself in the center.
3. Slowly raise the legs separately towards the inside of the bathtub, settling correctly on the seat in a position parallel to the bathtub itself.

4. Stand facing forward with extended legs. Keep hands always away from the seat, folding parts and mechanisms, while the lift is in motion.
5. Lower the lift so people can submerge themselves in the water.
6. To exit, follow the reverse procedure described above.

How to use the remote control

- Make sure the battery is charged.
- Press the "arrow down" button to lower the person lift. Press the "arrow up" button to raise it. If you wish to recline the backrest again, continue to hold the "arrow down" button after the seat has reached its lowest position. Once the desired angle is reached, release the button.
- To raise the backrest, press the "up arrow" button.
- Continue to press to raise the seat to its original position at the level of the tub.
- Orange button: lock button. During the ascent / descent by pressing the orange button you can stop the movement.
IMPORTANT!

While using the lift, a light will be visible on the remote control.

Green / blue light: the battery is charged.

Red light: the battery is almost empty.

How to remove the lift from the bathtub.

- Empty the bathtub completely from the water.
- Fold the flaps over the seat.
- Lift the lift through the front handle. Gently tilt the lift through the front handle.

NOTE

- For new batteries the first charge must be at least 8 hours.
- Recharge the batteries immediately after each use. Do not recharge for more than 24 hours.
- Disconnect the batteries from the lift if not used. Keep in a dry place.
- Use only the supplied battery charger.
- To ensure a long life, recharge once a month if the appliance is not used for long periods.

MAINTENANCE

For patient safety it is recommended to have a periodic inspection carried out by the manufacturer or by an authorized laboratory, at least once a year, to verify its suitability for use. Periodic inspection means a visual examination (in particular of the load-bearing structure of the lift and the lifting mechanism with the relative attachments, controls, safety devices and support devices of the person) and any functioning test and maintenance interventions that may be deemed necessary.

CLEANING

- It is advisable to wash the lifter after use to keep in perfect hygienic conditions and to avoid accumulation of dirt that can cause unexpected defects.
- Do not submerge the entire actuator in water to avoid the risk of damaging it.

GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal ecological islands for the envisaged recycling operations of the materials used.

TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / CE)

BATTERY REGISTRATION NUMBER IT21040P00006953

The battery contained in the appliance must not be disposed of as urban waste (Directive 2006/66 / CE), as it is subject to separate collection to avoid damage to the environment.

The user must therefore give the exhausted battery-accumulator free of charge to the appropriate municipal centers for the separate collection of electrical and electronic waste, or deliver it to the special containers provided by the points of sale.

To remove the battery: disconnect the remote control-battery from the rear cable.

WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

RESOLUTIONS OF PROBLEMS

SYMPTOMS	PROBABLE CAUSES	SOLUTIONS
----------	-----------------	-----------

No noise from the engine the lift does not work	-The battery is not installed correctly. -The battery is dead. - The contacts are dirty. - The emergency stop button has been pressed. - The control unit is damaged.	-Re-install the battery. - Charge the battery. -Clean contacts. - Press the button. Emergency stop. - Call the supplier to replace the control unit.
The lift can go up but not down	-Low battery. - Internal repairs required.	- Charge the battery. - Call the supplier for assistance.
The lift stopped during the elevation	- The load is excessive. - Required internal repairs.	- Check if the maximum load has been exceeded. - Call the supplier for assistance.
The battery does not charge	- The battery is defective. - Internal repairs required.	-Replace the battery. - Call the supplier for assistance.
The lift slips	-The suction cups are old or damaged.	- Replace the suction cups.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.