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Model PO100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to provide optimal support for the cervical spine during sleep. Thanks to its anatomical profile and the use of innovative thermoforming memory foam, the pillow adapts perfectly to the contours of the user's head and neck, regardless of sleeping position (on the side or back), effectively reducing pressure points, improving comfort and preventing blood circulation disorders. The product is particularly recommended for people struggling with back pain, neck pain, shoulder pain, arm numbness, snoring, insomnia or nervous tension, as well as anyone concerned with a healthy and restorative sleep; in addition, the pillow is made of hypo-allergenic materials that prevent the growth of micro-organisms and allergens, making it safe for allergy sufferers.

Intended use

A medical device designed to support the correct alignment of the cervical spine during sleep and to relieve pain and muscle tension in the neck and neck area.

Contraindications

Allergy to any of the product ingredients. If an allergic reaction occurs, discontinue use.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Made from innovative thermoforming memory foam that moulds to the user's head and neck.
- The foam is hypo-allergenic, does not absorb dust or moisture and prevents the growth of dust mites, micro-organisms and allergens.
- The anatomical, contoured shape provides optimum support regardless of sleeping position (on the side or back).
- The soft fabric cover, zipped up, allows for washing and hygiene maintenance over a long period of time.

First use

1. Carefully remove the cushion from its packaging and remove any protective film.
2. Optionally, remove the cover and wash it as recommended.
3. Once dry, put the cover back on the cushion.
4. Place the pillow on the bed so that the contoured part is under the neck.
5. Lie back on the pillow, allowing it to conform to the shape of your head and neck.
6. Observe your body's reaction during the first few days of use - a period of adaptation to the new shape of the cushion is possible.

Safety and usage rules

- Only use the pillow for its intended purpose as a medical device for cervical support during sleep.
- Wash the cover according to the manufacturer's recommendations before first use.
- Arrange the head in the middle of the pillow so that the neck is not overly bent or stretched; place the contoured section under the neck.
- Sleeping on the stomach is not recommended as it does not provide the correct support for the spine.
- Wash the cover regularly to maintain hygiene and prevent the growth of allergens and micro-organisms.
- Do not use the cushion if you have an allergic reaction to any of the ingredients.
- If discomfort or pain persists, consult your doctor or physiotherapist.
- Do not expose the cushion to high temperatures or chemically clean it to avoid damaging the shape memory foam.
- The product is OEKO-TEX Standard 100 certified, confirming the absence of substances harmful to health.

Maintenance

- The cushion cover can be washed by hand or in a washing machine at up to 30°C using a mild detergent and fabric softener; do not use bleach.
- Dissolve the detergent before adding it to the wash to avoid localised discolouration of the fabric.
- Wash separately from other garments to avoid staining.
- Do not soak the cover in cold water for more than 5 minutes.
- Dry on the left side, avoiding direct sunlight.
- After washing, it is advisable to stretch and hang the cover to dry; the ironing temperature should not exceed 110°C.
- The pillow insert (foam core) is not suitable for washing, dry-cleaning, bleaching or exposure to direct sunlight. Store in a dry and well-ventilated place, preferably in the original packaging, to prevent dampness and mould growth.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 2 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Batch/series number
	Model		Warning
	Date of production		Medical device manufacturer
	Unique identification code of the device		Keep out of sunlight
	Protect from moisture		

    	Cover: Wash at up to 30°C, do not bleach, dry flat in the shade, do not iron, do not dry-clean.
    	Pillow core: Do not wash, do not bleach, dry flat, do not iron, do not dry-clean.

Product content:

Knitted polyamide/elastane (also fleece), viscose/polyester/elastane, polyester/cotton/elastane and polyester/elastane, white and ready-made dyed (acid, reactive and disperse dyes), 100% polyester (also coral fleece), white and dyed with disperse dyes, partly with white polyurethane foam, including accessories (sewing thread, zip, plastic snap/clasp, printed and woven label, elastic band); Pillow core made of thermoplastic polyurethane foam (memory foam).

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ACTION S.A.