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**Model: SP300AM**

#### Wstęp

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**Model SP300AM**

### **Introduction**

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

### **Description of the medical device**

Medical device equipment (beds: medical / rehabilitation / nursing) designed to assist elderly, disabled and convalescent patients in getting up and into bed. The handrail is made of height-adjustable chrome-plated steel, equipped with a padded pocket for small items and non-slip handles covered with soft foam for comfort and safety. The device is characterised by quick and simple installation, it is placed between the mattress and the bed frame and is designed to provide stability and support for users weighing up to 130 kg, minimising the risk of falls and re-injury during daily bed use activities.

### **Intended use**

The bed rail is designed for use in hospital wards, rehabilitation centres and in home care for patients who require support with bed mobility, especially the elderly, after surgery or with restricted mobility. The device is used in the prevention of falls and as a supportive element in the rehabilitation process of patients recovering from injuries, orthopaedic or neurological surgery.

## Contraindications

The bed rail must not be used for patients with memory problems and disorientation who may try to walk over the rail, and for patients with uncontrolled body movements and agitation. Use in patients with a tendency to climb is also contraindicated, which can lead to more dangerous falls from greater heights.

## Assembly

1. Place the handrail frame on a flat surface and lay out all the components.
2. Extend the telescopic base arms to the correct length to match the width of the bed.
3. Slide the lower part of the frame under the mattress so that the vertical handrail protrudes at the edge of the bed.
4. Adjust the height of the handle to suit your individual needs.
5. Attach the stabilising strap (if supplied) to the bed frame or frame and tighten it firmly to prevent the handrail from moving during use.
6. Check the stability of the assembly, making sure that the handrail is well seated between the mattress and the frame and does not move when pressure is applied.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

**Maximum permissible user weight – 130 kg.**

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

## Safety and usage rules

- Fit the handrail according to the instructions, making sure that the entire base is under the mattress and the stabilising strap is firmly tightened; the handrail should fit tightly to the mattress without gaps forming.
- Regularly check the stability and fixing of the railings, especially after every bed rearrangement or bedding change.
- Avoid gaps greater than 60 mm between the end of the handrail and the headrest and greater than 120 mm between the handrail and the surface of the mattress to prevent the risk of wedging or strangulation.
- Do not use extra cushions, blankets or other materials to fill gaps, as these can compress and increase the risk of wedging.
- The handrail should not be used by people with cognitive impairment who may attempt to walk over the handrail, or as the sole means of preventing falls in patients at high risk of falling.
- Always make sure the handrail is stable and does not move under pressure before standing up or sitting down; ask your carer for extra support if necessary.

- Do not use the handrail on waterbeds or in situations where the design of the bed prevents its stable attachment.
- Regularly inspect the condition of the handrail, looking out for foam wear, cracks and damage to structural components.

### **Maintenance**

1. Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
2. Avoid the use of: strong detergents, abrasives, bleach, petrol and all solvents, as these may cause damage to the product, visible defects and lead to a loss of warranty.
3. Use only soft cloths or sponges to clean and maintain the steel components, avoiding sharp brushes and rough materials that can damage the protective surface of the stainless steel and promote corrosion.
4. Regularly inspect the condition of the handrail, paying attention to any signs of mechanical damage such as cracks, deformations or loose parts and, if these occur, immediately withdraw the product from use and contact the service centre or manufacturer.

### **Storage and transport conditions**

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

### **Disposal**

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

### **Product life**

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

### **Warranty**

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

### **NOTE**

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

### Product content

- Chrome-plated steel (frame and support structure).
- Polyurethane foam (handle finish for comfort and slip resistance).
- Plastic (connecting elements, plugs, adjustments).
- Nylon strap (stabilising element for securing the handrail to the bed).

### Technical data

- Maximum permissible weight load: 130 kg
- Product dimensions: 94 cm × 51 cm
- Net weight: 2.1 kg

### Explanation of symbols

|                                                                                     |                                                                         |                                                                                      |                                                                             |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------|--------------------------------------------------------------------------------------|-----------------------------------------------------------------------------|
|    | Conformity of the device with EU Regulation 2017/745 on medical devices |    | Medical device                                                              |
|    | Take a look at the instructions for use                                 |     | Maximum permissible weight (indicated on the label and in the instructions) |
|  | Model                                                                   |  | Batch/series number                                                         |
|  | Date of production                                                      |   | Warning                                                                     |
|  | Unique identification code of the device                                |   | Medical device manufacturer                                                 |
|  | Protect from moisture                                                   |   | Keep out of sunlight                                                        |

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**ACTION S.A.**