AT51049
INSTRUKCJA UŻYTKOWANIA, PNEUMATYCZNA WANNA DLA
NIEPEŁNOSPRAWNYCH
INSTRUCTION MANUAL, PNEUMATIC BATHTUB FOR THE DISABLED
GEBRAUCHSANWEISUNG, PNEUMATISCHE BADEWANNE FÜR MENSCHEN
MIT EINGESCHRÄNKTER MOBILITÄT
NÁVOD K OBSLUZE, MOBILNÍ KOUPACÍ VANA
NÁVOD NA OBSLUHU, PNEUMATICKÁ VAŇA PRE POSTIHNUTÝCH
MANUEL D'UTILISATION, BAIGNOIRE PNEUMATIQUE POUR HANDICAPÉS
HANDLEIDING, PNEUMATISCH BAD VOOR GEHANDICAPTEN
MANUAL DE INSTRUCCIONES, BAÑERA NEUMÁTICA PARA DISCAPACITADOS
MANUALE D'ISTRUZIONI, VASCA PNEUMATICA PER DISABILI
BRUKSANVISNING, PNEUMATISKT BADKAR FÖR FUNKTIONSHINDRADE

PL

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Thank you for purchasing our product. Please carefully read the instruction manual before use. Do not use the product before you have read and understood this manual. If you do not understand the warnings, notes, and recommendations, please contact a healthcare specialist or the seller to avoid damaging the product or personal injury.

NOTE: Check all parts for damage that may have occurred during transportation. If such damages are observed, DO NOT use the product. More information is available from the manufacturer.

TARGET PATIENT GROUP

This product is intended for individuals dealing with diseases, dysfunctions, or injuries for treatment/rehabilitation or compensation (see the section regarding the intended use of the product in this manual). The product can be purchased by the user independently or based on the recommendations of a doctor, therapist, or other specialist. Both in the case of independent purchase and when prescribed by a doctor/therapist/other specialist, it is necessary to consider the available sizes/necessary features and variants of the product, indications and contraindications for use, as well as information provided by the manufacturer.

APPLICATION/INDICATIONS

This unique bathtub is designed to simplify the work of a person caring for a bedridden patient or an elderly person. The bathtub allows not only to wash the user in bed without wetting the bedding but also provides maximum comfort and convenience during hygienic procedures. To avoid possible injuries to the user or damage to the equipment, adhere to the recommendations contained in this manual and general safety requirements.

SAFETY MEASURES

Before use, ensure that the bathtub is in a stable horizontal position.

Do not allow the bathtub to come into contact with electrical devices and cables, and ensure that there are no sharp, piercing, etc., objects nearby that could breach the integrity of the tank.

To prevent air leaks and emptying of the tank before use, ensure that the check valve is tightly closed.

Do not overfill the bathtub to avoid splashing or spilling water.

Do not cover the air pump's inlets/outlets; they must always be well-ventilated.

Use the appropriate nozzle when operating the compressor. The compressor must be protected from impacts and falls.

To avoid electric shock, it is not recommended to connect the compressor using an extension cord. During the compressor's operation, do not allow small objects to enter its interior.

Do not use or store the compressor in high humidity rooms.

Keep away from children.

Remember to maintain an optimal, safe water temperature, one that will not burn or expose the patient to cold.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product. Acute infections - any acute inflammatory states and skin infections.

Open wounds - presence of wounds, ulcers, or other skin injuries.

Heart diseases - heart failure, arterial hypertension, etc.

Respiratory diseases - asthma, chronic obstructive pulmonary disease (COPD).

Neurological disorders - epilepsy, certain neurological conditions.

Circulatory issues - thrombosis, varicose veins, peripheral circulation disorders.

DESCRIPTION

Bathtub - inflatable, made of vinyl. The bathtub is equipped with a water drainage hose. A compressor is used to inflate and deflate the air from the bathtub's tank.

TECHNICAL DATA

Dimensions of the bathtub when inflated (length x width x height) mm: 1900x900x220

Mains voltage: 220V

Power consumption: no more than 150W

Maximum load: 130 kg

Compressor operating mode: no longer than 20 minutes with a 15-minute break.

The compressor is not designed for continuous operation. The compressor is equipped with two universal nozzles.

This symbol indicates the maximum user weight.

COMPONENTS: Pneumatic bathtub for the disabled, compressor, patches and glue, instruction manual

PREPARATION FOR USE

To inflate the bathtub, you must: Connect the nozzle to the compressor outlet to pump air. Open the valve cover, insert the compressor nozzle firmly into the valve connector and press the switch button to the "I" position to turn on the compressor. Ensure that the bathtub is inflated, then press the switch to the "O" position, remove the nozzle, and tightly close the valve cover, then disconnect the compressor. Never block the airflow through other ventilation holes. Always use the appropriate nozzle to quickly inflate the item. The bathtub is ready to use.

To deflate the bathtub, you should: Open the bathtub's deflation valve. Attach the compressor nozzle to the deflation valve and slide the switch to the "I" position to turn on the compressor. Ensure that no air remains in the bathtub, press the switch to the "O" position, disconnect the compressor, and disconnect the compressor.

WARNING! Before inflating the bathtub, ensure the user is placed inside it. Consider the load capacity of the bed on which the bathtub will be installed.

USAGE

The user must lie on a bed covered with a waterproof sheet without clothes when washing the body. Place the deflated bathtub on the bed under the user, turning them from side to side. Inflate the bathtub and fill it with water. Wash the user. Deflate the bathtub, release the bathtub from the user by turning it. Remove the bathtub from the bed, re-inflate the bathtub, wash and thoroughly dry it.

**INFORMATION:**

The set includes patches and glue which may be useful in case of product damage. The glue-coated patch should be applied over the damage. The patch should be glued to a dry surface. **WARNING:** In the event of a "serious incident" related to the product that directly or indirectly led, could lead, or may lead to any of the following events:

- a) death of the patient, user, or another person, Or
- b) temporary or permanent deterioration of the health condition of the patient, user, or another person, or
- c) serious danger to public health, such a "serious incident" must be reported to the manufacturer and the relevant national authority in the country in which the user or patient resides. In the case of Poland, the relevant authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

WARNING: In case of pain, allergic reactions, or other troubling, unclear symptoms related to the use of the medical device, consult a healthcare professional.

WARNING: It is prohibited to use the product in any way other than in accordance with its intended purpose.

CLEANING

Do not use solvents (acetone, etc.) or abrasive materials. Wipe the bathtub and its parts dry after disinfection. Leave the bathtub inflated for several hours for complete drying. Then deflate the bathtub, fold it, and store the product. **STORAGE** The product should be stored in a dry place, not exposed to direct sunlight. The bathtub in the manufacturer's packaging should be stored in closed rooms at a temperature of +5 °C to +40 °C and relative humidity up to 80%. **WARNING!** The mode of operation of the compressor should not exceed 20 minutes with a 15-minute break. The compressor is not designed for continuous operation. Transportation and storage of the bathtub without the manufacturer's packaging does not guarantee its safety. Damages to the bathtub resulting from transport or storage without the manufacturer's packaging are repaired by the consumer.

DISPOSAL

The bathtub does not contain harmful substances and components that pose a risk to human health and the environment during and after the period of use and during disposal. Disposal of the bathtub is carried out separately according to material groups in accordance with the relevant regulatory documentation. This product contains electronic parts. After the product's lifespan has ended, do not throw it away in the regular trash. To dispose of the product properly, it should be taken to an appropriate collection point for electrical and electronic equipment. Proper disposal prevents potential damage to the environment and human health according to applicable legal regulations regarding the recycling of electrical and electronic equipment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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