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Model: KP400AM

Wstęp

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Model KP400AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed for people with reduced mobility who have difficulty performing hygiene tasks independently in the shower or bathroom. It is equipped with a swivel seat that allows it to turn comfortably without having to stand up, making it easy to wash and transfer. The lightweight aluminium construction ensures stability and safety, while the adjustable height allows the stool to be adapted to the individual user's needs. Non-slip feet increase safety when using the product in wet conditions, and the overall product is easy to clean and designed for use in both domestic and healthcare settings.

Intended use

The medical device is intended to facilitate the performance of hygiene-related at home and in hospital, especially for people with reduced mobility. It can be used both in the shower and in the bath, providing stability and comfort while bathing.

Contraindications

People with balance problems, coordination disorders, spinal instability or contraindications to

sitting should avoid using the product due to the risk of falling and aggravating their condition.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- **Seat** Made of durable plastic, round and fitted with a swivel mechanism to allow free rotation during use. The seat has drainage holes to allow water to drain away, and there are cut-outs on the sides to act as handles, making it easier to carry the stool and to aid sitting down and standing up.
- **Backrest** It has no backrest, which makes it easier to use the stool in the limited space of the shower and allows access from either side.
- **Frame / Legs:** The structure is made of lightweight, corrosion-resistant aluminium. The legs are straight, topped with rubber non-slip pads for stability and safety on wet surfaces.
- **Height adjustment:** The stool's legs have visible holes and a telescopic design, which allows the seat height to be infinitely adjustable according to the user's individual needs.
- **Connectors:** The components attaching the seat to the frame are made of metal or durable plastic, ensuring the durability and safety of the entire structure.

Assembly

1. **Preparation of elements:** Unpack all the parts of the stool and place them on a flat surface. The kit should include a seat, four legs, fasteners.
2. **Leg fixing:** Slide each leg into the corresponding slots located under the seat. Push the legs in until a distinctive click is heard or until the adjustment holes align.
3. **Height adjustment:** Set the desired seat height by adjusting the length of each leg using the telescopic mechanism and locking it by inserting the pins or turning the knobs in the appropriate places.
4. **Stability check:** Make sure all components are securely fastened and the stool stands firmly on all four legs. Check that the rubber pads on the feet are well seated and do not slip on the floor.
5. **Rotating mechanism test:** Before first use, check that the seat rotates smoothly and without resistance.

Safety and usage rules

- Before each use, check the stability of the stool and ensure that all components are correctly fixed and that the swivel mechanism operates smoothly.
- Place the stool on a level, stable surface of the shower tray or shower floor, making sure that all the feet are touching the floor and the rubber pads are clean and undamaged.

- The maximum permissible load on the product should not be exceeded.
- The user should sit down slowly, holding on to the handles (if present) and making sure that the stool does not move when sitting down and standing up.
- Take extra care when turning on the seat to avoid losing your balance.
- After each use, the stool should be rinsed with clean water and dried to prevent sediment build-up and to maintain hygiene.
- Regularly check the condition of the stool, including the swivel mechanism, fixings and rubber leg ends, and if damage or wear is noticed, discontinue use until the components are repaired or replaced.
- The stool is intended for use in the shower or bathroom only, in accordance with the manufacturer's instructions.
- Do not use the stool on slippery or unstable surfaces outside the bathroom.
- The stool should only be used by the persons for whom it is intended, with safety precautions and under the supervision of a carer in the case of persons with reduced mobility or balance.

Maintenance

- Regular cleaning of the chair should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- It is a good idea to check the condition of structural components at least once every three months, including fixing bolts, as well as pipes for any cracks. In addition, it is advisable to inspect the rubber cap regularly; if it is found to be worn, it should be replaced with a new one.

Storage and transport conditions

The stool should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Seat and back: HDPE plastic (high density polyethylene) - waterproof, anti-slip material, easy to disinfect
- Frame and legs: anodised aluminium - lightweight, stainless, corrosion-resistant
- Leg caps: rubber or TPE - non-slip to prevent sliding on wet surfaces
- Fasteners: stainless steel or high-strength plastic

Technical data

- Adjustable height: approx. 37-49 cm
- Seat diameter: approx. 33-40 cm
- Overall dimensions: approx. 49 × 41 cm
- Maximum user weight 120 kg

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ACTION S.A.