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INSTRUCTIONS FOR USE

Issue number and date: 1 / 07.2025

High pulpit rehabilitation walker

Model B200AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to assist the upright standing and mobility of people with reduced mobility. The product is designed for safety, comfort and ergonomic use during rehabilitation, recovery from injury or surgery, as well as in the daily functioning of older people. It features a lightweight and stable height-adjustable design, is equipped with comfortable underarm supports and four swivel castors with brakes for comfort, safety and ease of use both at home and in medical facilities.

Intended use

The product is intended to support upright standing and mobility of persons with reduced mobility, in particular patients undergoing rehabilitation after trauma, surgery, stroke or other neurological and orthopaedic conditions, as well as elderly persons having difficulty walking

independently. The product enables safe movement, provides stability and support during learning to walk or returning to physical activity, relieving pressure on the lower limbs and reducing the risk of falls. It should be used as directed by medical personnel, in home, healthcare or rehabilitation settings.

Contraindications The medical device should not be used by people with severe balance disorders or significant loss of muscle strength that prevents safe use of the device without constant support from a third person. Inability to move independently and serious health conditions that may increase the risk of a fall or injury when using a walker are also contraindications. The device should be selected individually with the help of a specialist or physiotherapist, and a doctor should be consulted if there is any doubt about the possibility of safe use.

User

The product can be operated independently by the user. A product designed for the layman.

Description of product construction

Fig. 1 Elements of product construction

No.	Component name	Description of functions
1	Armpit support	Soft, contoured armpit support section for user comfort and stabilisation.
2	Main frame	Supporting structure made of metal to ensure the stability of the walker.
3	Adjustment handles	Knobs for adjusting the height of the armpit supports to suit the user's height.
4	Shelf/seat	Wide section to support body weight as a seat, increasing comfort.

No.	Component name	Description of functions
5	Wheels with brake.	Four swivel castors for easy mobility; often fitted with a locking device (brake) for safety.
6	Stabilising bar	Horizontal tube to reinforce the structure and increase the rigidity of the frame.

Preparation for use

1. Remove the walker from the packaging and check that all components are complete.
2. Fold out the frame - this model has a foldable, lightweight frame for easy transport and storage.
3. Fix the handles (if removable) and ensure that all components are securely locked.
4. Adjust the height of the handles - set them at the height of the user's wrists when standing upright at the walker. Correct adjustment ensures safety and comfort.

How to use

1. Place the walker on a flat surface.
2. Stand between the back legs of the walker, holding the handles with both hands.
3. Move the walker slightly forward and then take a step, moving first one, then the other leg.
4. Repeat the activity, moving the walker in front of you a short distance and following it.
5. Don't lean your whole body weight on the walker - it's for balance, not for lifting your body.
6. It is recommended to use the walker on dry and level surfaces to avoid slipping.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

- Before each use, check the stability and condition of all components of the walker.
- Adjust the height of the armpit rests for the forearms to suit the height of the user.
- Ensure that all locks and adjustment mechanisms are properly locked.
- Only use the walker on even, dry and stable surfaces.
- Move slowly by moving the walker in front of you with both hands and then getting your legs.
- Don't rest your whole body weight on the walker - it is for support and assistance, not for lifting your body.
- If you need to stop, lock the wheels with the brakes.

- Do not use a walker if you have a severe balance disorder or cannot move independently.
- Avoid obstacles, stairs and slippery surfaces when using the product.
- Clean the walker regularly and check the condition of the tips and wheels, replacing them if they wear out.
- Use the walker according to the recommendations of your doctor or physiotherapist and the manufacturer's instructions.

Maintenance and cleaning

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Maintaining the cleanliness of all components is key to maintaining the functionality of the product; they must be stored in appropriate conditions. Storage in rooms with high humidity (e.g. baths, laundries) or outdoors, combined with a lack of hygiene, can lead to a rapid loss of functional and aesthetic properties.
- Avoid exposing the walker to extreme weather conditions (rain, snow, intense sun), as some parts may change colour.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Frame: aluminium or powder-coated steel
- Grips: plastic (e.g. polyurethane or EVA foam) with non-slip properties.
- Wheels: plastic (PVC or rubber)
- Control elements and connectors: steel, plastic

Technical data:

- Maximum load: 120 kg
- Adjustable height: approx. 78-100 cm (depending on version).
- Design: Foldable, easily transportable
- Number of wheels: 4, swivel, with brakes
- Purpose: for home use and medical facilities
- Product weight: approx. 7-10 kg (depending on version)
- Finish: scratch- and corrosion-resistant coating

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ACTION S.A.