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Model: SW300AM

Wstęp

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Model SW300AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device designed to make it easier for elderly people with limited mobility or disabilities to get in and out of the bathtub safely. The product has a stable, sturdy construction and is equipped with a security railing to provide additional support and a sense of balance during use. The step increases the comfort and safety of daily hygiene activities, minimising the risk of slips and falls thanks to mats and anti-slip pads. Its lightweight design allows it to be easily carried and can also be used as a platform to reach objects from higher shelves, making it a versatile support in the daily functioning of people with mobility problems.

Intended use

A medical device designed to facilitate safe entry into and exit from the bathtub for people with reduced mobility, providing stable support and minimising the risk of slipping during daily hygiene activities.

Contraindications

Do not use with imbalance, coordination disorders, weight over 120 kg, step damage or slippery ground.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 120 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Description of product construction

- Metal frame - the step stool structure is based on solid, chrome-plated tubes that provide strength and stability during use.
- Step surface covered with an anti-slip mat - the upper part of the platform is equipped with a black grooved mat, which effectively prevents feet from slipping.
- Four widely spaced legs - guarantee even distribution of the user's weight and increase the stability of the entire structure.
- Legs finished with rubber pads - each leg has non-slip rubber feet to protect the floor from scratches and prevent the step from sliding.
- No sharp edges - all elements of the step have rounded shapes, making it safer to use.
- Security handrail - a high, ergonomically shaped handrail facilitates ascent and descent, providing additional support and safety.
- Compact dimensions - the step stool is small and lightweight, making it easy to carry and store.

Assembly

- Remove all step components from the packaging and check for completeness.
- Place the base frame on a flat surface, making sure the four legs are stable.
- Attach the security railing to the step frame by sliding it into the appropriate mounting holes in the base.
- Screw the handrail to the frame using the screws or clamps provided (usually no tools are required, but use a spanner or screwdriver if necessary).
- Check that the handrail is firmly attached and does not wobble.
- Make sure all legs are finished with non-slip rubber pads.
- Place the stepladder in its intended place of use and check its stability before first use.

Safety and usage rules

- Before each use, make sure the step stool stands firmly on a flat, dry surface and that all rubber pads on the legs are clean and not slipping on the ground.
- Always use a handrail when ascending and descending a step - it provides extra support and balance, minimising the risk of falling.
- Do not exceed the maximum permissible load specified by the manufacturer.
- Regularly check the condition of the non-slip matting on the step surface and the pads on the feet - damage or wear to these components can significantly reduce safety.

- Before climbing a step, make sure there is no water, soap or other slippery substances on the step that could cause you to slip.
- Once you have finished using it, you can easily move the step stool to another location - the lightweight design makes it easy to move, but always check the stability once it has been set up in the new location.
- Do not use the step stool if you notice any structural damage, loose screws or cracks, in which case stop using it immediately and consult the manufacturer or retailer.
- It is recommended that people with very limited mobility use the step stool in the presence of a carer, especially when first attempting to use it.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Wipe the step down dry after each use to prevent moisture build-up and deposits.
- Regularly check the condition of the rubber leg caps and adjustment elements - replace them with new ones if worn or damaged.
- Avoid prolonged contact of the step stool with water - do not leave it in the bath or on a wet surface for long periods of time.
- Store the step stool in a dry place, away from direct sunlight and heat sources.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

Product content:

- Frame and handrail: chromium-plated steel (or powder-coated steel) for high mechanical strength, corrosion resistance and aesthetic appearance.
- Step surface: anti-slip plastic, usually polypropylene (PP) with fluting for increased grip.
- Leg pads: rubber or other flexible non-slip material to protect the floor from scratches and prevent the step stool from sliding.
- Mounting components: steel (screws, clamps), sometimes with the addition of plastics for easier assembly and corrosion protection.

Technical data

- Step/platform height: approx. 23 cm
- Platform dimensions: width 36 cm, depth 28 cm
- Handrail height (from the ground): 87 cm
- Leg spacing: 35 x 41 cm
- Maximum load: 120 kg
- Product weight: approx. 3.5 kg

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ACTION S.A.