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## INSTRUCTIONS FOR USE

Issue number and date: 1 / 07.2025

### **Off-road aluminium rehabilitation walker**

#### **Model CH200AM**

#### **Introduction**

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

#### **Description of the medical device**

The off-road aluminium rehabilitation walker is a specialised piece of medical equipment designed for people with reduced mobility who need stable support when moving, especially on uneven surfaces and off-road. The off-road walker is particularly recommended for older people and anyone who needs stable support during outdoor activities and more difficult terrain. Its design and functionality significantly enhance the safety and comfort of daily mobility.

#### **Intended use**

The off-road aluminium rehabilitation walker is designed to support people with limited mobility who require stable support when walking, especially on uneven surfaces. It enables safe uprighting, learning to walk and independent movement for the elderly persons, after injury, orthopaedic surgery or during neurological rehabilitation. It prevents falls, increases the user's

independence and facilitates daily functioning both indoors and outdoors. The use of the walker should be according to the recommendations of the doctor or physiotherapist.

**Contraindications**

The medical device should not be used by people with severe balance disorders or significant loss of muscle strength that prevents safe use of the device without constant support from a third party. Inability to move independently and serious health conditions that may increase the risk of a fall or injury when using a walker are also contraindications. The device should be selected individually with the help of a specialist or physiotherapist, and a doctor should be consulted if there is any doubt about the possibility of safe use.

**User**

The product can be operated independently by the user. A product designed for the layman.

**Description of product construction**

**Fig. 1 Elements of product construction**

| No. | Element             | Description                                                                           |
|-----|---------------------|---------------------------------------------------------------------------------------|
| 1   | Handles with brakes | Ergonomic handles with built-in brakes for user safety.                               |
| 2   | Aluminium frame,    | Lightweight, robust design for stability and ease of transport.                       |
| 3   | Seat                | A comfortable place to rest while walking.                                            |
| 4   | Backrest            | Soft strap to provide back support while sitting.                                     |
| 5   | Off-road wheels     | Four wheels with an increased diameter, making it easier to navigate uneven surfaces. |

| No. | Element                      | Description                                                                               |
|-----|------------------------------|-------------------------------------------------------------------------------------------|
| 6   | Shopping bag                 | Practical, roomy bag attached under the seat to carry personal belongings.                |
| 7   | Folding mechanism            | A system that allows the walker to be folded quickly and easily for transport or storage. |
| 8   | Height adjustment of handles | The height of the handles can be adjusted to suit the height of the user.                 |

### Preparation for use

1. Unpack the walker and check that all components are complete and undamaged.
2. Unfold the frame of the walker according to the instructions, making sure that the folding mechanism is correctly locked.
3. Fit a shopping bag and, if included, a cane or crutch holder.
4. Adjust the height of the handles to suit your height and secure the adjustment.
5. Check the function of the brakes and their locking mechanism, as well as the stability of the seat and backrest before first use.

### How to use

1. Stand straight behind the walker, holding the handles with both hands.
2. Make sure the brakes are unlocked before you start walking.
3. Slowly move the walker in front of you, taking steps in its wake each time you move.
4. If necessary, stop, lock the brakes and sit on the seat, leaning against the backrest.
5. After use, unlock the brakes, carefully stand up, grasp the handles and continue walking.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

**Maximum permissible user weight – 136 kg.**

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

### Safety and usage rules

- Always check that the walker is correctly assembled and that all components (including brakes, wheels and handles) are in working order and securely fastened before use.
- Adjust the height of the handles so that they are level with the wrists of a person standing upright - the arms should be slightly bent at the elbows when holding the walker.
- Always use the walker on a stable, dry and level surface; avoid slippery surfaces, carpets, thresholds and other obstacles that can cause loss of balance.

- When moving, move the walker in front of you, then take a step forward, transferring your body weight to the device; do not lean too hard to avoid knocking the walker over.
- Always lock the brakes before sitting down on the seat, and after resting make sure they are unlocked before continuing to walk.
- Do not carry heavy objects in the bag that could disturb the stability of the walker.
- Regularly check the condition of the walker, in particular the brakes, wheels and folding mechanism, and use a service if necessary.
- If you have any doubts or your condition deteriorates, consult your doctor or physiotherapist before continuing to use the walker.

### **Maintenance and cleaning**

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- Maintaining the cleanliness of all components is key to maintaining the functionality of the product; they must be stored in appropriate conditions. Storage in rooms with high humidity (e.g. baths, laundries) or outdoors, combined with a lack of hygiene, can lead to a rapid loss of functional and aesthetic properties.
- Avoid exposing the walker to extreme weather conditions (rain, snow, intense sun), as some parts may change colour.

### **Storage and transport conditions**

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

### **Disposal**

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

### **Product life**

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

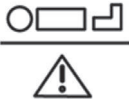
### **Warranty**

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

## NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

## Explanation of symbols

|                                                                                     |                                                                         |                                                                                     |                                                                      |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------|-------------------------------------------------------------------------------------|----------------------------------------------------------------------|
|    | Conformity of the device with EU Regulation 2017/745 on medical devices |   | Medical device                                                       |
|    | Take a look at the instructions for use                                 |    | Maximum user weight (indicated on the label and in the instructions) |
|    | Model                                                                   |   | Batch/series number                                                  |
|    | Date of production                                                      |    | Warning                                                              |
|   | Unique identification code of the device                                |   | Medical device manufacturer                                          |
|  | Protect from moisture                                                   |  | Keep out of sunlight                                                 |

## Product content:

- Aluminium frame
- Wheels: plastic (PVC)
- Seat and back: synthetic fabric (mesh or nylon)
- Handles: plastic with non-slip coating
- Bag: synthetic fabric

## Technical data:

- Overall width: 66 cm
- Handle height (adjustable): 79.5 cm to 92.5 cm
- Wheel diameter: 20 cm
- Seat depth: 25 cm
- Seat width: 47 cm
- Seat height: 53 cm
- Width when folded: 29 cm
- Unit weight 7.6 kg
- Maximum user weight 136 kg
- Foldable: yes
- Braking system: manual, with adjustable and interchangeable brake cables
- Components requiring regular maintenance: wheels, handles, brake cables, brake lever, bearings, knobs, bag, bolts and nuts
- Cleaning instructions: clean with a damp cloth or sponge with warm water and soap; household antibacterial products may be used; do not use rough sponges or scrapers

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**ACTION S.A.**