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Model: P200AM

Wstęp

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Model P200AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

A medical device that assists in positioning a person lying in bed, especially for the elderly, patients requiring rehabilitation or long-term care. The product allows the angle of the backrest to be adjusted in several positions, providing comfortable and safe back support when resting, eating, reading or performing other activities in a semi-sitting position, which significantly improves comfort and supports the rehabilitation process.

Intended use

Therapeutic positioning and stabilisation of the spine in immobilised patients and prevention of immobilisation-related complications such as bedsore syndromes and circulatory disorders. Indicated in the rehabilitation of patients after spinal injuries, surgery, neurological diseases and in geriatrics.

Contraindications

Contraindications include unstable spinal fractures, acute inflammatory conditions, severe spinal deformities, fresh surgical wounds and patients with impaired consciousness unable to signal discomfort. The device should also not be used for people exceeding a maximum load of 100 kg and in patients with severe respiratory failure, where repositioning may worsen the clinical condition.

Assembly

1. Remove the backrest from the packaging and lay it out on a flat surface.
2. Position the backrest frame so that the cushion section is at the top.
3. Fold out the side armrests, tilting them outwards to a horizontal position.
4. Position the lower backrest (thigh support) flat on the bed.
5. Rest your back against the main part of the backrest, and push the whole thing against the headrest of the bed.
6. Adjust the angle of the backrest using the adjustment mechanism at the back (visible as a rack-and-pinion) - set the desired tilt level.
7. Ensure that all components are securely locked and the backrest is ready for use.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 100 kg.

Any use not in accordance with the intended use is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

1. Check the stability and correct distribution of all backrest components before use.
2. Make sure the device is placed on a flat, stable bed surface.
3. Check that the angle adjustment mechanism is correctly locked before seating or positioning the patient.
4. Do not exceed the maximum permissible load specified by the manufacturer.
5. Do not use the product if it is damaged, deformed or has visible signs of material wear.
6. The backrest should only be used for its intended purpose - to position and support the patient's back.
7. Do not leave children or people with reduced mobility unsupervised when using the backrest.
8. Regularly check the condition of the product and keep it clean by following the manufacturer's cleaning recommendations.
9. Do not sit or stand on the armrests or use the product as a step or platform.
10. If you experience discomfort, pain or other distressing symptoms during use, stop using the device immediately.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.

- Disinfect items in contact with the patient's body in accordance with the recommendations for medical devices, using agents approved for disinfection of medical equipment
- Thoroughly dry all parts after each cleaning to prevent mould and corrosion.
- Periodically check the condition of the frame, adjustment mechanism and armrests - if you notice damage or play, do not use the product until repaired.
- Do not use aggressive chemicals, solvents or abrasives for cleaning.
- Check fixings and bolts regularly, tightening them as necessary.
- If the textile is permanently soiled or damaged, consult the manufacturer/distributor about replacing the item.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Product content

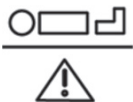
- Frame: powder-coated steel or aluminium (lightweight, robust design)
- Sheathing: polyester or nylon fabric (easy to clean, tear-resistant)
- Armrests: plastic with soft foam or textile covering
- Head rest: polyurethane foam covered with polyester fabric
- Connecting elements and adjustment mechanism: steel / aluminium
- End caps and small fittings: plastic

Technical data

- Tilt angle adjustment: 5-6 levels (rack and pinion mechanism)
- Backrest width: 54.5 cm

- Backrest height: 46 cm
- Maximum angle of inclination: 70°
- Maximum permissible user weight – 100 kg.
- Product weight 2.7 kg

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum permissible weight per tabletop (indicated on the label and in the instructions)
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer
	Protect from moisture		Keep out of sunlight

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ACTION S.A.