# VERMEIREN

Laly

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

### **Instructions to the specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: B, 2016-12

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© Vermeiren GROUP 2016

|           |                                          |
|-----------|------------------------------------------|
| <b>EN</b> | <b>Contents</b>                          |
| 1.        | Technical details .....2                 |
| 2.        | Components .....2                        |
| 3.        | Explanation of symbols .....2            |
| 4.        | Check after delivery .....2              |
| 5.        | Intended use .....2                      |
| 6.        | Before use .....2                        |
| 7.        | Mounting / Removing backrest .....3      |
| 8.        | Folding / Unfolding the chair .....3     |
| 8.1.      | Folding shower chair .....3              |
| 8.2.      | Unfolding shower chair .....3            |
| 9.        | Seat height adjustment .....3            |
| 10.       | Carrying the shower chair .....3         |
| 11.       | Safety instructions .....4               |
| 12.       | Maintenance, inspection, cleaning .....4 |

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### Foldable shower chairs "Laly"

#### 1. Technical details

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

|                                                                                                  |                                                                                        |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------|
| <b>Make</b>                                                                                      | Vermeiren                                                                              |
| <b>Address</b>                                                                                   | Vermeirenplein 1/15,<br>B-2920 Kalmthout                                               |
| <b>Type</b>                                                                                      | Foldable shower chair                                                                  |
| <b>Model</b>                                                                                     | Laly  |
| Width                                                                                            | 540 mm                                                                                 |
| Length                                                                                           | 580 mm                                                                                 |
| Height                                                                                           | 910 mm - 1010 mm                                                                       |
| Dismantled width                                                                                 | 520 mm                                                                                 |
| Dismantled length                                                                                | 790 mm                                                                                 |
| Dismantled height                                                                                | 120 mm                                                                                 |
| Seat height                                                                                      | 430 mm - 530 mm                                                                        |
| Seat width                                                                                       | 410 mm                                                                                 |
| Seat depth                                                                                       | 410 mm                                                                                 |
| Backrest height                                                                                  | 500 mm                                                                                 |
| Weight total                                                                                     | 3,80 kg                                                                                |
| Patient weight                                                                                   | <b>max. 130 kg</b>                                                                     |
| We reserve the right to introduce technical changes.<br>Measurement tolerance +/- 15 mm / 1,5 kg |                                                                                        |


First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the shower chair, read the instruction manual carefully: it will familiarise you with this product.  
If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

#### 2. Components

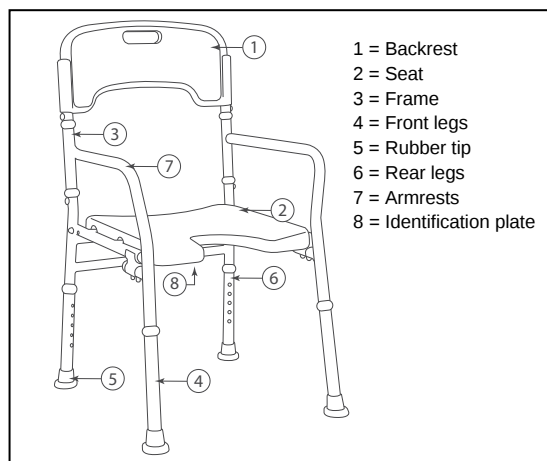#### 3. Explanation of symbols

Maximum mass

CE conformity

Type designation

#### 4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Seat
- Backrest
- Height adjustment legs
- 4 Rubber tips
- Armrests
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

#### 5. Intended use

The shower chair shall be used by 1 person.  
The shower chair shall be used indoor, in the shower, bathroom.

All our shower chairs are ideal for usage in the shower, bathtub, bathroom or other humid environments. The specially designed rubbers at the end of each leg will create a safe stability for the chair and avoid the sliding of the chair.

The construction of the chair allows full use by elderly people or during revalidation in home environment.

The chair shall not be used as toy for children.  
The chair should only be used on horizontal, non smooth surfaces.


The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

#### 6. Before use

- Remove all packaging before using the shower chair for the first time.
- Mount backrest.
- Unfold the chair.
- Adjust the seat in the desired height.

## 7. Mounting / Removing backrest

**⚠ CAUTION: Risk of injury - Check that the backrest is firmly fixated.**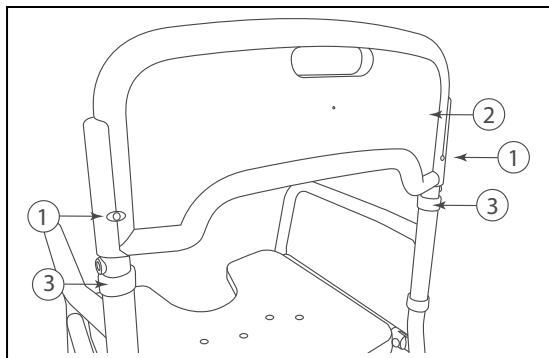

Mounting backrest:

- Mount the backrest ② on the two backrest tubes ③ until the spring pins ① clicks in the hole.
- Check that the backrest is firmly fixated.

Removing backrest:


- Press on the two spring pins ① on the rear of the backrest.
- Move the backrest ② upwards out the backrest tubes ③.

## 8. Folding / Unfolding the chair

**⚠ WARNING: Risk of injury - Check that the chair is completely unfold before using it.**

**⚠ WARNING: Risk of clamping - Keep fingers, buckles and clothes away from the entrapment zone.**

To transport the chair in the car or airplane you can fold the chair.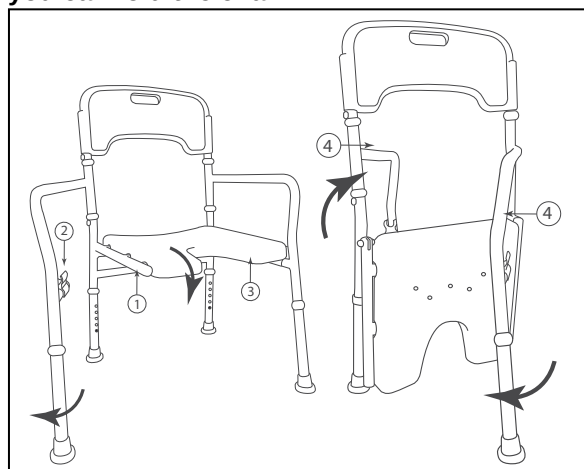

### 8.1. Folding shower chair

- Lift the seat tubes ① out the two plastic clips ②.
- Push the seat ③ downwards.
- Fold one side frame ④ frontwards against the seat.
- Fold the other side frame ④ backwards against the seat.

### 8.2. Unfolding shower chair


- Fold both side frames ④ to the side.
- Pull the seat ③ upwards.
- Push the seat tubes ① in the two plastic clips ②.

## 9. Seat height adjustment

**⚠ CAUTION: Risk of falling - Secure the seat firmly before using the shower chair. Check that the spring pin ① is firmly fixated in a hole.**

**⚠ CAUTION: Risk of falling - Do not exceed the maximum seat height adjustment, your shower chair will not be stable anymore.**

**⚠ CAUTION: Risk of tipping over - Place the four legs at the same height.**

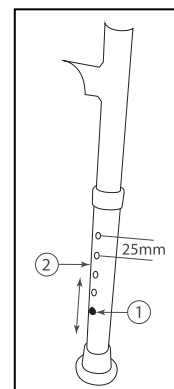Before using the shower chair you must adjust the seat in the correct height.

The seat can be adjusted in height by following steps:

- Press on the spring pin ①.
- Move the leg ② until the desired height (5 different positions: steps from 25 mm).
- Check that the spring pin ① is clicked back and firmly fixated in a hole.
- Adjust the other legs with the same instructions.
- Check that all legs are placed in the same position.

## 10. Carrying the shower chair

The best way to carry the shower chair: grasp the handgrip of the backrest or grasp the two armrests.

## 11. Safety instructions

- **Do not exceed the maximum user weight. The maximum user weight for your chair is 130 kg.**
- Use the shower chair on horizontal, non-smooth surfaces, all four legs must have contact with the ground.
- Keep it away from children. The shower chair shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- Risk of burns – Be careful when using in extreme hot or cold environments for a sufficient amount of time and when touching.
- Do not use the shower chair when it is not working properly.

- Secure the seat firmly before using the shower chair.
- Check that the shower chair is completely unfold before using it.
- Do not lift the shower chair when the user is seated on the chair - Risk of injury or damage.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

## 12. Maintenance, inspection, cleaning

For the maintenance manual of the shower chair refer to the Vermeiren website:  
[www.vermeiren.com](http://www.vermeiren.com).

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim tis warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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### **Vermeiren Suisse S.A.**